

Assembly


⚠ Caution: Before using the Pressure Fit System, the top plate, bottom plate, supporting posts and adjustable track must be visually checked to ensure that there are no missing parts or unusual wear and tear. Top plate and Bottom plate should be cleaned and dry with wet cloth before installation (See Cleaning Rubber Feet Instructions). Should anything look unusual contact your local dealer prior to use.

Failure to comply with this caution could result in serious injury to the operator, the individual being lifted and/or damage to the Pressure Fit and/or portable lift unit.

Assembling the Support Posts



Step 1:
Placed the bottom foot on the floor. Insert the bottom plate through the opening located at the bottom of the supporting post.

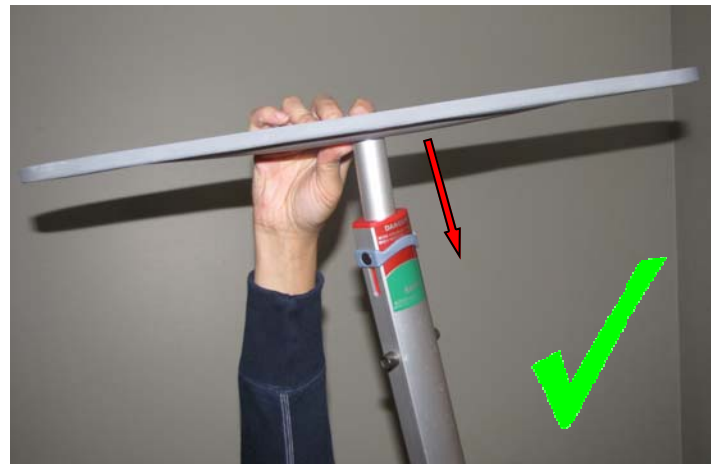

Step 2:
Tilt the supporting post to the side to facilitate the installation of the top foot. Insert the top plate through the top opening of the supporting post.



NOTE:
Never install Top Foot at the Bottom section of Post



NOTE:
Never install Bottom Foot at the Top section of Post

Assembly

Assembling the Support Posts (Continued)




Step 3a:
Press the red button down to release the safety lock.



Step 3b:
Safety Lock Released.



Step 4a:
Grab and Push down on the top section of post with two hands as shown in Picture.

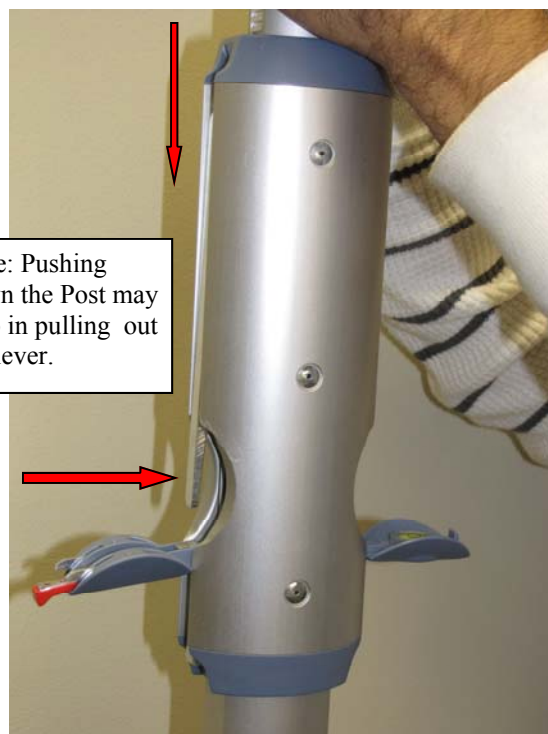

Step 4b:
While pushing with two hands, the lever will move out as shown in picture.
Note:
This will help in pulling out the lever.

Assembly

Assembling the Support Posts (Continued)



Step 5a:
Press in and hold the Ratchet while pulling out the lever .



Step 5b:
Push the lever up to engage with the magnet.



Step 6:
Push the top section of post up towards the ceiling with Top foot. Upon contact with the ceiling, continue pushing up until you hear one or two more “clicks”.

Note:
At this point, the post should be exerting pressure between the floor and ceiling.



Step 7:
Swing open post level. Please ensure the level is fully open.

Assembly

Assembling the Support Posts (Continued)



Step 8a:
Grasp bottom section of post, lift slightly and adjust until bubble is centered and post is perpendicular to floor and ceiling.



Step 8b:
Bubble centered.



Step 9a:
Pull the lever down until it “clicks” into place.



Step 9b:
Press the lever in to secure it.
Note: Lever must rest in vertical position.

Assembly

Assembling the Support Posts (Continued)



Step 10:
Ensure Pressure Gauge is in the Safe Zone.
Note: Green must be visible on both sides.



Step 11a:
Swing back the safety lock and level to the original position.



Step 11b:
Safety Lock and Level must be firmly Engaged in housing.

Step 12:
Install the second post repeating steps 1 through 11.
Note:
The distance between the two posts should be 7 to 10 feet.


Mounting the Adjustable Track




Step 13a:
Hook one end of the track to post pin.



Step 13b:
Extend the track and hook to the other post.

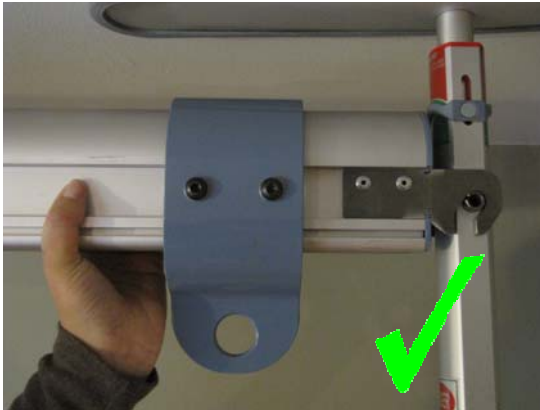

Note: Ensure track connector is sitting properly on the spring pin as shown above.

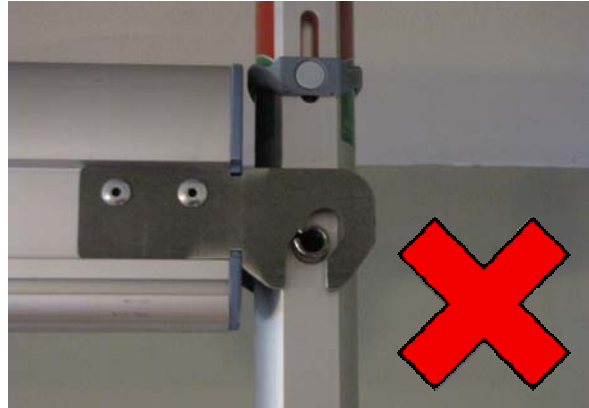

Note: Don't leave the track connector on the spring pin as shown above.



Step 14a:
Attach the track end cap as shown above.



Step 14b:
Install the other end cap.

Final Inspection Prior to Use




Ensure Pressure Gauge is in the Safe Zone.

Note: Green must be visible on both sides.



Safety Lock and Level must be firmly Engaged in housing.

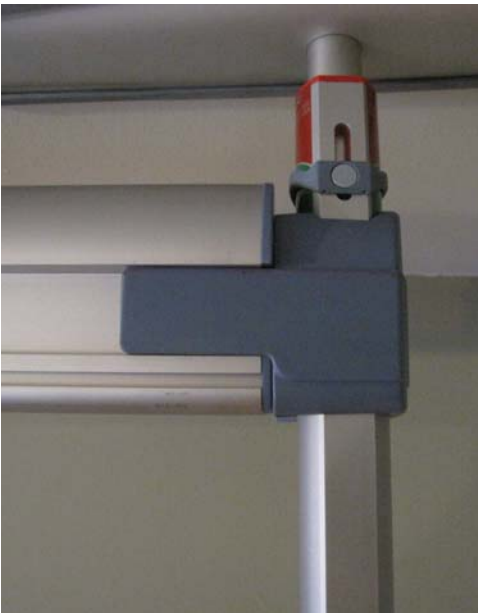

End covers are installed properly

Completed Pressure Fit System



Completed Pressure Fit System with a M-400 lift.

Mounting the Portable Lift

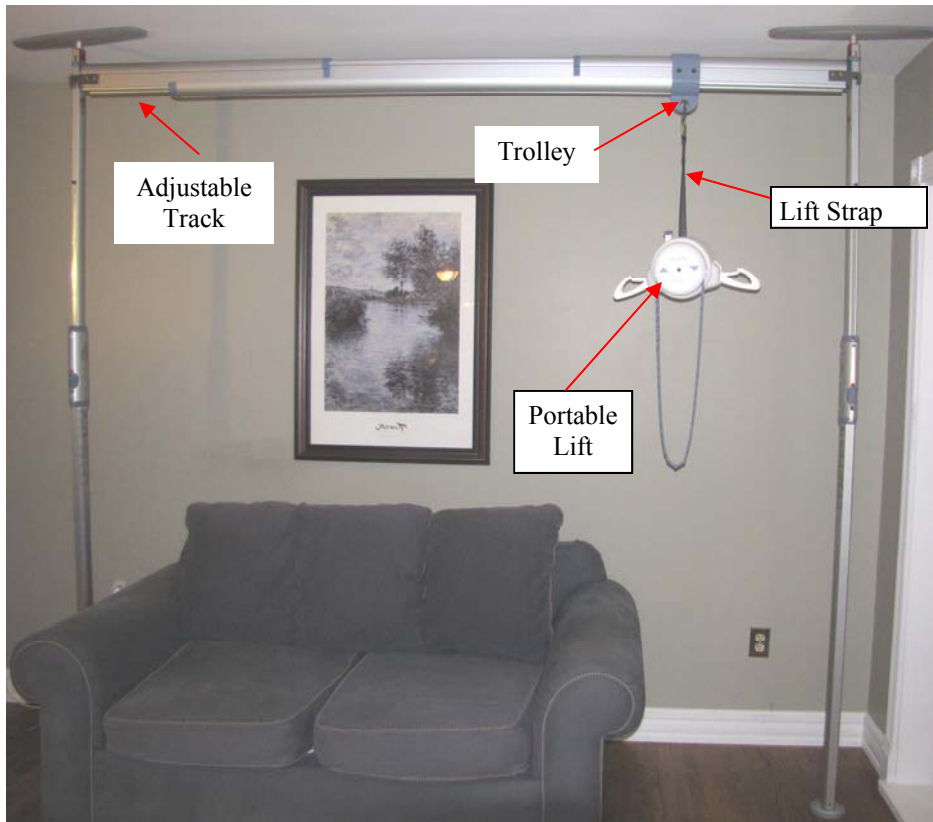


Figure 1A– FULLY ASSEMBLED PRESSURE FIT SYSTEM

Prior to mounting the portable lift onto the trolley of the Pressure Fit System, please read the owners manual of the portable lift. Be sure that the instructions on the use of the lift and any accessories, such as slings are thoroughly understood before attempting to use them with the Pressure Fit System.

FAILURE TO COMPLY WITH THIS MAY RESULT IN INJURY TO THE INDIVIDUAL BEING LIFTED AND/OR THE CAREGIVER, OR DAMAGE TO THE LIFT AND/OR THE PRESSURE FIT SYSTEM.

Following the instructions for the portable lift, attach the carabiner, (or other similar attaching device) located at the end of the lift strap, to the hole of the Pressure Fit trolley. Be sure that the carabiner is securely attached to the trolley prior to proceeding with the transfer.

FAILURE TO COMPLY WITH THIS MAY RESULT IN INJURY TO THE INDIVIDUAL BEING LIFTED AND/OR THE CAREGIVER, OR DAMAGE TO THE LIFT AND/OR THE PRESSURE FIT SYSTEM.

Proceed to transfer the individual in the manner described in the owners' manual for the portable lift and sling. When the transfer is completed the lift may be removed from the trolley.