
Overview & Quick Reference Guide

A1CNow® Self Check Test System

Note: All instances of the box or pouches in this insert are representative ONLY. Please refer to the packaging included with your specific kit.

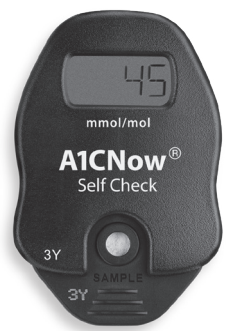
+1-317-870-5610

We invite you to call us and we will guide you through the test.

IMPORTANT!

- FIRST**, we strongly recommend you read all instructions carefully and watch the brief instructional video on our website at ptsdiagnostics.com. **THEN**, follow all steps as you perform the test.
- Do not open pouches** until instructed.
- Make sure lot number on analyzer matches lot numbers on pouches.
- Use indoors between 18°C - 25°C / 64°F - 77°F and out of direct sunlight.
- Do not use if product is past the expiration date.
- If the kit has recently been kept at temperatures outside the recommended conditions, keep the kit at room temperature for at least one hour before use.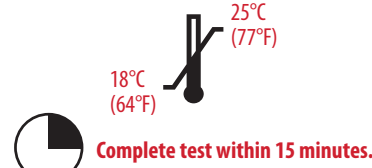


Complete test within 15 minutes.

6 Open Foil Cartridge Pouch* AFTER Blood Collection

- Insert cartridge into analyzer immediately and use within 2 minutes.
- Tear foil pouch open at the notches on the sides.

Caution: Do NOT open foil pouch until this step. Use within 2 minutes of opening. If foil pouch is damaged, do not use.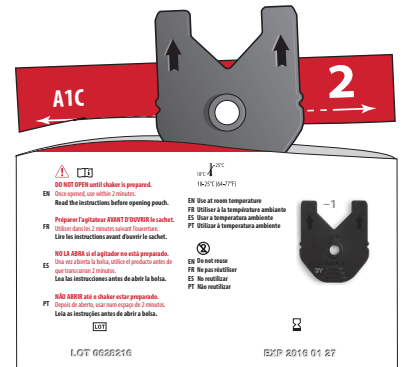


- DO NOT adjust your medication unless instructed to do so by your doctor or healthcare provider.**
- If you have any questions about your A1C result, please contact your doctor or healthcare provider.

Caution: This device contains material of animal origin and should be handled as a potential carrier and transmitter of disease.


For additional information, see the reverse side of this insert.

If you are not able to access the instructional video on our website, we encourage you to call customer service at +1-317-870-5610 to walk you through the test.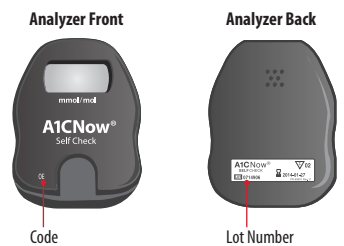

Match Lot Numbers

Use analyzer only with the materials included in the original kit. The analyzer will expire after the programmed number of tests have been run. If another test cartridge is inserted, the analyzer will display "00TL."

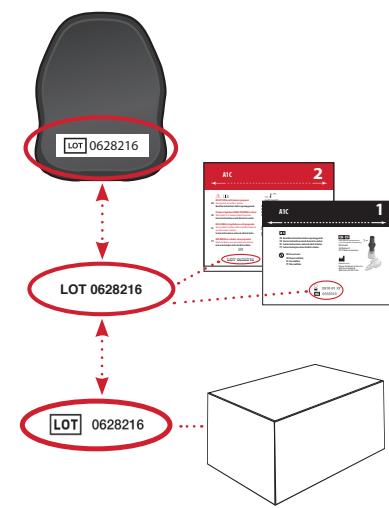
7 Insert Cartridge

- "Click" test cartridge into place.
- Analyzer and test cartridge codes must match.
- If codes do not match, call Customer Service at +1-317-870-5610.**


8 Prepare Shaker

Remove shaker base.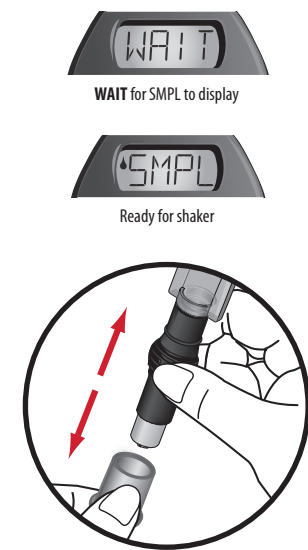

1 Open Plastic Shaker Pouch*

Do NOT open foil cartridge pouch until Step 6.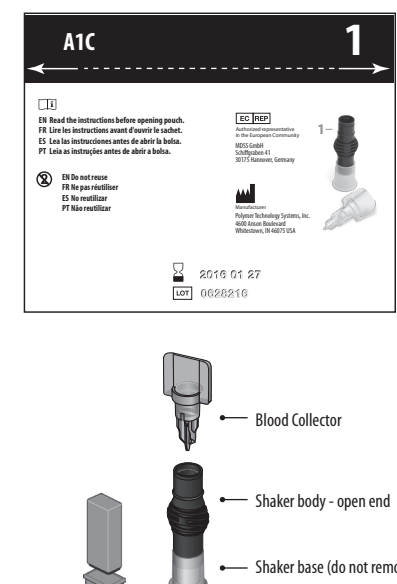

9 Dispense Sample into Cartridge


- Ensure analyzer is on level surface.
- Push down completely to dispense diluted sample and remove in a continuous motion.
- Remove sampler from cartridge quickly.
- "RUN" will appear.

Do not handle analyzer again until test is complete!

2 Prick Finger


- Wash hands or clean with alcohol and dry before testing.
- Remove cap and press lancet into finger.
- Gently squeeze finger to get adequate blood.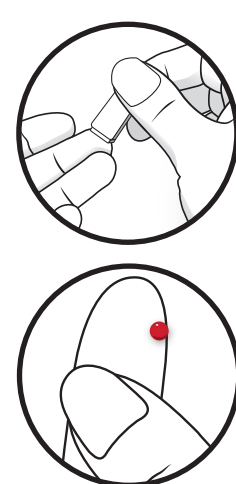

Actual size required

3 Collect Blood

- Gently touch blood collector to top of blood drop to fill.
- Make sure collector is adequately filled.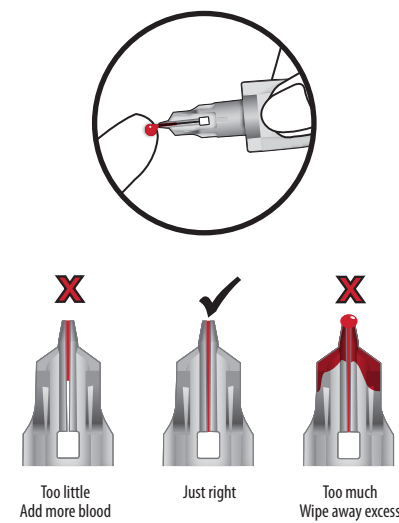

Too little Add more blood Just right Too much Wipe away excess

11 Dispose of Cartridge / Save Analyzer


- Dispose of cartridge in your household waste.
- Save analyzer for additional tests. Analyzer will display, for example, "01TL" if there is one test left and "00TL" if you just used your last test.
- Record your A1C result in the result log in Step 12.
- Analyzer will show result for 15 minutes and will turn off automatically.

NOTE: To run another test, use a new shaker, blood collector, test cartridge, and lancet from the same kit and return to Step 1.

4 Insert Blood Collector


- Fully insert blood collector into shaker body.
- Push hard while twisting to fully insert.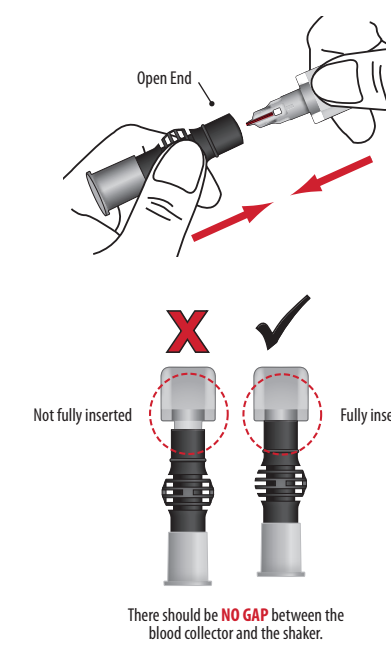

There should be **NO GAP** between the blood collector and the shaker.

5 Mix

- Shake vigorously 6-8 times (about 5 seconds). This will mix the blood with the solution.
- Stand shaker on table while preparing cartridge.

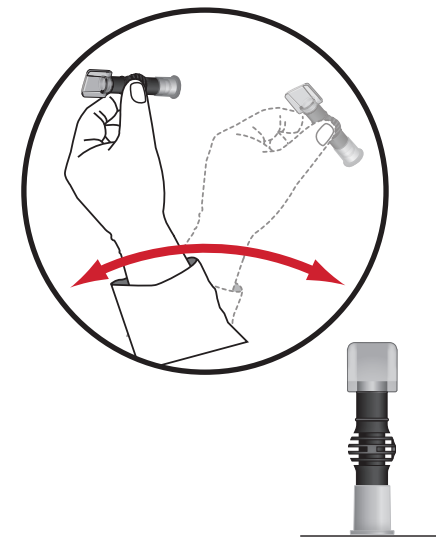Result Interpretation

- Do not take any decision of medical relevance without first consulting your healthcare practitioner.**
- When this device is used for monitoring of an existing disease, adapt the treatment **only** if you have received the appropriate training to do so.

12 Log Result

Chart your progress here.

Record your A1C results here, and bring this with you when you see your doctor. Mark down your A1C goal in the space provided.

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