



# Sofia<sup>2</sup> Flu+SARS Antigen FIA

AVAILABLE FOR SALE IN THE USA AS EUA

## Flu A, Flu B + SARS-CoV-2 at Your Fingertips

One Test, Three Results, 15 Minutes



Accurate, objective and automated results in **15 minutes**



Track via **secure, remote instrument management** with **Virena<sup>®</sup>**



**Excellent performance** compared to viral culture or molecular methods



**Integrated data management** automatically stores test and user history



**Flexible, dual mode testing** for high throughput in variety of laboratory environments




User and patient ID captured with **onboard barcode scanner**




# Sofia 2 Flu + SARS Antigen FIA - Procedures

## Nasal/Nasopharyngeal Swab Procedure

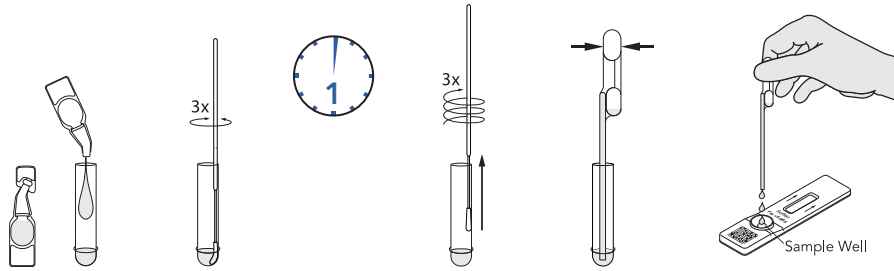


## Sofia 2 Development Modes

### WALK AWAY Mode – Walk away and multitask

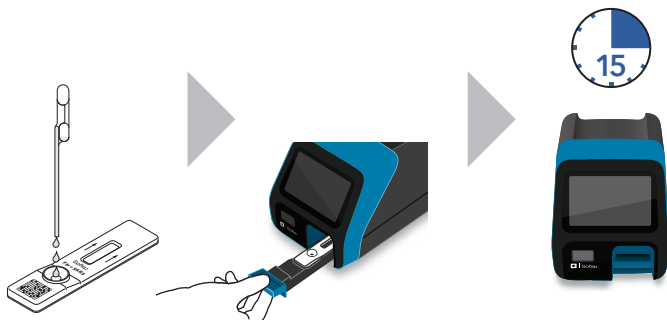

### READ NOW Mode – Batch multiple samples per hour

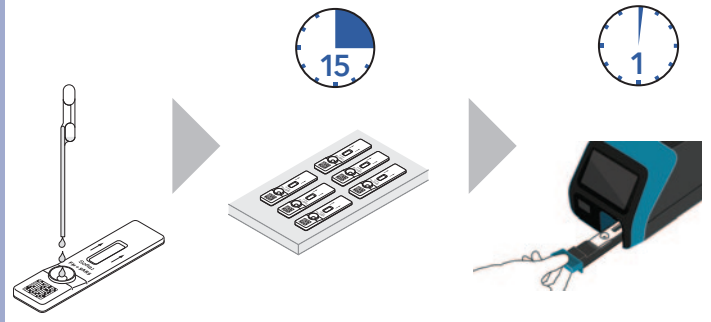

## Sofia 2 Flu + SARS Antigen FIA\*

- Nasal and nasopharyngeal specimens
- Accurate detection with direct samples
- Results in 15 minutes
- Kit includes 25 sterile nasal or nasopharyngeal swabs and a set of positive and negative control swabs
- Room temperature storage
- 12-month shelf life from date of manufacture



## Sofia 2 Flu + SARS Antigen FIA Clinical Performance\*

	Sample Type	Sens/PPA*	Spec/NPA*
<b>Influenza A</b>	Nasal	90%	95%
	Nasopharyngeal	97.1%	94.6%
<b>Influenza B</b>	Nasal	89%	96%
	Nasopharyngeal	90%	97%
<b>SARS-CoV-2*</b>	Nasal	95.2%*	100%*

\*Results for SARS-CoV-2 are expressed as PPA/NPA and were tested on nasal swabs in the study, however, may also be used with NP swabs.

Refer to the Package Insert for additional performance claims.

Sofia 2 Flu + SARS Antigen FIA – 25 Test Kit (nasal swabs): Catalog #20377

25 Test Kit (nasopharyngeal swabs): Catalog #20390

