

EASYPPOINT®

retractable needle

- AUTOMATED NEEDLE RETRACTION
- EFFECTIVELY REDUCES THE RISK OF NEEDLESTICK INJURIES
- EASY TO USE - REQUIRES MINIMAL TRAINING
- ACTIVATION CAN BE ACHIEVED IN A VARIETY OF WAYS
- MAY BE ACTIVATED BEFORE OR AFTER REMOVAL FROM PATIENT
- PRE-REMOVAL ACTIVATION REDUCES THE RISK OF EXPOSURE TO THE CONTAMINATED SHARP
- ACTIVATION OF SAFETY MECHANISM AUTOMATICALLY COVERS SYRINGE LUER TIP, REDUCING RISK OF EXPOSURE TO MEDICATION, BLOOD/BODY FLUIDS
- REMAINS SAFE THROUGH DISPOSAL

compatible with both luer lock & luer slip syringes



may be used with pre-filled syringes



EASYPPOINT®

RETRACTABLE NEEDLES

(50 per box/400 per case)

Now available in
NEW
sizes

Catalog # Needle Size

82211	30G x ½"
82091	25G x 5/8"
82011	25G x 1"
85011	25G x 1 ½"
82031	23G x 1"
85021	22G x 1"
85031	22G x 1 ½"
85041	21G x 1"
85051	21G x 1 ½"
85061	20G x 1"
85071	20G x 1 ½"
85081	18G x 1"
85091	18G x 1 ½"

OTHER AVAILABLE SAFETY PRODUCTS

VanishPoint® Syringe
VanishPoint® Blood Collection Tube Holder
VanishPoint® Blood Collection Set
VanishPoint® IV Catheter
Patient Safe® Syringe



511 Lobo Lane, Little Elm, TX 75068-5295

P 972 • 294 • 1770 TF 1 • 888 • 703 • 1010 F 972 • 294 • 4400

rtiservice@vanishpoint.com



©2015, 2020 Copyright Retractable Technologies, Inc. All rights reserved.

M098 REV.6 EN, FEB 2020

EASYPPOINT®

retractable needle



— EASY TO USE —


- minimal change in injection technique
- allows for single-handed activation
- **clinician's hands and fingers remain behind the exposed sharp** at all times




1
Attach EasyPoint needle to any luer lock or luer slip syringe. Administer medication according to institutional policy.



2
Apply pressure to the color-coded tab and the safety chamber using a squeezing motion, allowing the needle to automatically retract into the safety chamber. **This device is designed for activation before or after removing the needle from the patient.**

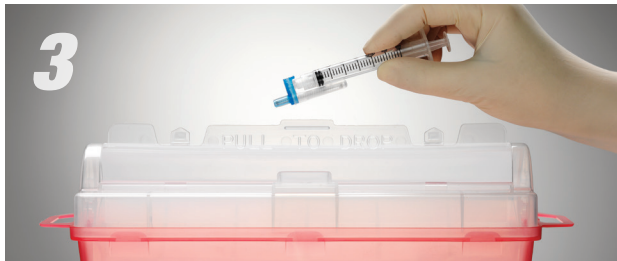

3
Dispose in an appropriate sharps container per protocol of institution.

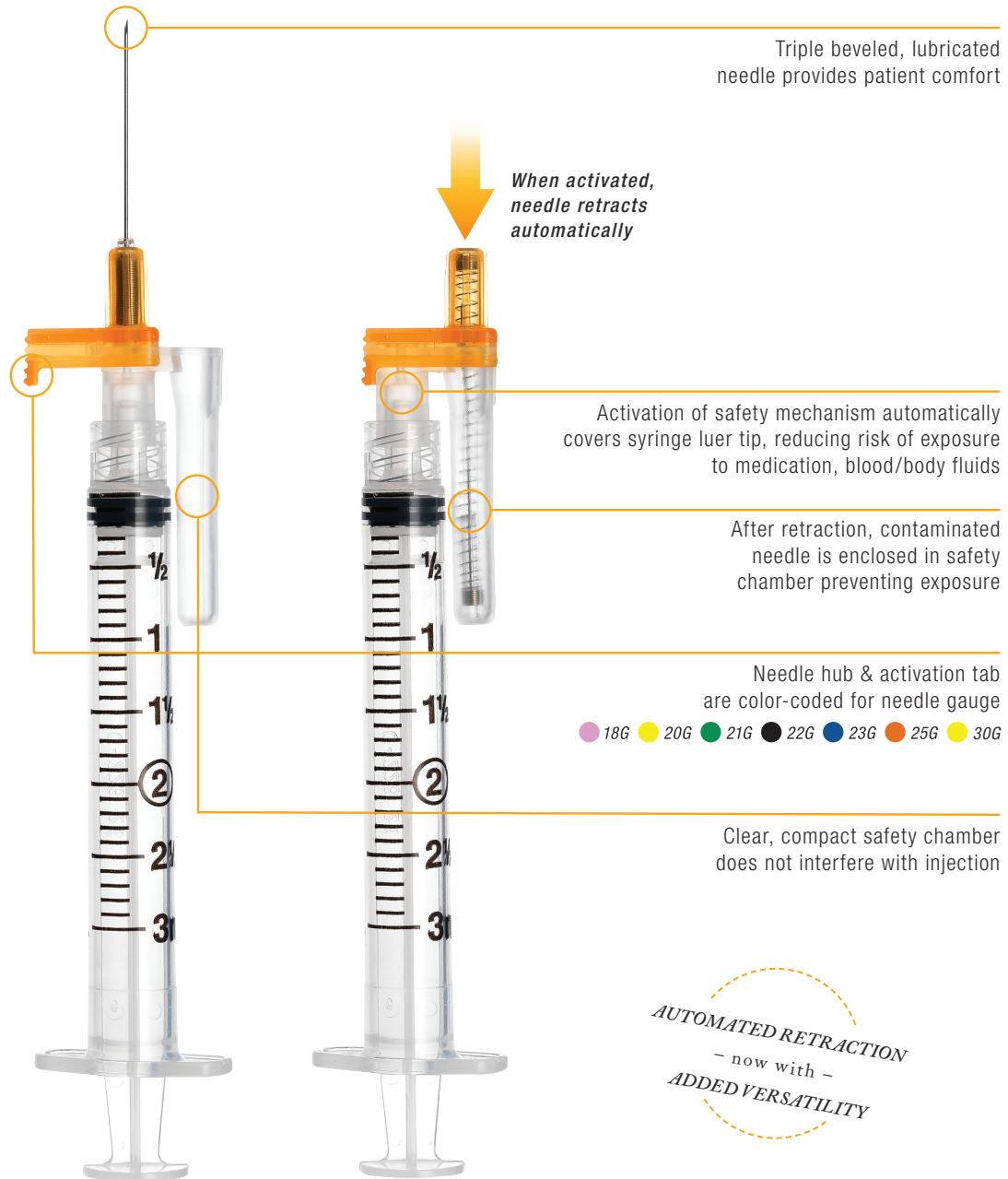

EASYPOINT®