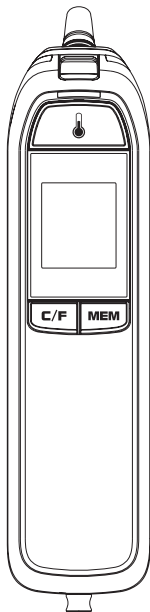
**ri-thermo<sup>®</sup> tymPRO**

**ri-thermo<sup>®</sup> tymPRO+**

**Tympanic Thermometer / Trommelfellthermometer**

**Thermomètre tympanique / Termómetro timpánico**

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# TABLE OF CONTENTS

<b>INTRODUCTION</b>	<b>2</b>
<b>INTENDED USE</b>	<b>2</b>
<b>HOW DOES IT WORK</b>	<b>2</b>
<b>CONTENTS</b>	<b>2</b>
<b>IMPORTANT SAFETY INSTRUCTIONS</b>	<b>3</b>
<b>CAUTIONS AND WARNINGS</b>	<b>4</b>
<b>APPEARANCE AND KEY FUNCTIONS</b>	<b>5</b>
<b>HINTS ON TAKING TEMPERATURES</b>	<b>5</b>
<b>DISPLAY SCREEN</b>	<b>6</b>
<b>REPLACING THE BATTERY</b>	<b>7</b>
<b>CHANGING UNIT OF MEASUREMENT</b>	<b>7</b>
<b>RESTRICTIONS OF USE</b>	<b>8</b>
<b>USING THE DEVICE</b>	<b>8</b>
<b>RECALLING PAST READINGS</b>	<b>10</b>
<b>BLUETOOTH PAIRING (ri-thermo® tymPRO+ ONLY)</b>	<b>11</b>
<b>ABOUT NORMAL BODY TEMPERATURE &amp; FEVER</b>	<b>12</b>
<b>MAINTENANCE</b>	<b>13</b>
<b>TROUBLESHOOTING</b>	<b>14</b>
<b>SYMBOL INFORMATION</b>	<b>15</b>
<b>SPECIFICATIONS</b>	<b>15</b>
<b>REFERENCE STANDARDS</b>	<b>16</b>

# INTRODUCTION

Thank you for choosing ri-thermo<sup>®</sup> tymPRO / ri-thermo<sup>®</sup> tymPRO+ tympanic thermometer.

Please read this instruction manual first, so you can use this thermometer safely and correctly. Please keep this instruction manual for future reference. This innovative medical device uses advanced infrared (IR) technology to measure temperature instantly and accurately.

Before using this product, please read the following contents thoroughly and carefully.

## INTENDED USE

ri-thermo<sup>®</sup> tymPRO / ri-thermo<sup>®</sup> tymPRO+ tympanic thermometer is intended for the intermittent measurement and monitoring of human body temperature from the ear canal. The device is intended for use of all ages for professional use by one with a good understanding of the operation instruction.

## HOW DOES IT WORK

The thermometer measures the infrared heat generated by the eardrum and its surrounding tissue.

The thermometer then converts it into a temperature value shown on LCD.

### **NOTE:**

The thermometer does not emit any infrared signal.

## CONTENTS

- Meter
- Operation Instructions
- Batteries
- Cradle
- 10 probe covers x 2

# **IMPORTANT SAFETY INSTRUCTIONS**

## **READ THIS BEFORE USING AND KEEP THESE INSTRUCTIONS**

The following basic safety precautions should always be taken.

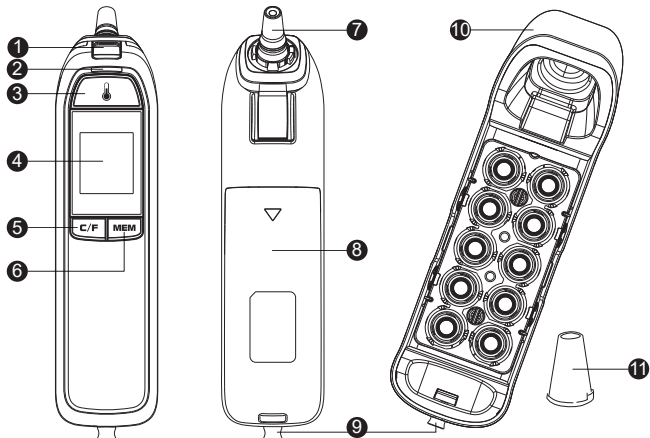
1. Close supervision is necessary when the thermometer is used by, on, or near children, or handicapped persons.
2. Use the thermometer only for the intended use described in this manual.
3. Do not use the thermometer if it is not working properly, or if it has suffered any damage.
4. Keep the sensor end clean and free of debris. See Maintenance section for instructions.
5. Do not use ethylene oxide gas, heat, autoclave, or any other harsh methods to sterilize the device.
6. If coming from an environment of warmer or cooler temperature or after a period of exertion, allow the user and the thermometer to acclimate to room temperature for 20 minutes prior to taking a measurement.
7. As the body site for measurement may be affected by sweat, oil and the surrounding temperature, the reading shall be taken as a reference only.
8. Do not use in presence of flammable anesthetic mixtures.
9. Do not use accessories which are not supplied or recommended by the manufacturer. Do not try to modify the device to prevent any dangers.
10. Proper maintenance is essential to the longevity of your device. This thermometer has been calibrated at the factory. If you follow the instructions, you don't need to adjust it regularly. However, if you are concerned about the accuracy of measurement, please contact the local customer service or place of purchase for help.
11. Always contact the manufacturer or the manufacturer's representative to report unexpected operation or event. Do not try to fix it by yourself.
12. When using the thermometer, stay away from electromagnetic radiation, such as the mobile in use.

13. Do not expose the device to strong electrostatic fields or strong magnetic fields to avoid affecting the measurement accuracy.
14. Use of this device adjacent to or stacked with other device should be avoided because it could result in improper operation.
15. Keep probe covers out of reach of children. The small parts detached from the device may result children choking from inhaling or swallowing.
16. Do not try to maintain the device while it is in use.

## **CAUTIONS AND WARNINGS**

- Always operate the thermometer in an operating temperature range 10°C to 40°C (50°F to 104°F), and relative humidity between 30% to 85%.
- Always store the thermometer in a cool and dry place: temperature between -20°C to 60°C (-4°F to 140°F) and relative humidity between 30% to 85%.
- Avoid direct sunlight.
- This thermometer is not intended to substitute for a consultation with your physician.
- Use only supplied or recommended probe covers by the manufacture.
- For proper hygiene, do not re-use probe cover. Damaged probe cover may result in error display.
- Keep probe covers out of reach of children.
- Temperature measurements may differ from left to right ear. Always take the temperature in the same ear.

# APPEARANCE AND KEY FUNCTIONS

- |                               |                           |
|-------------------------------|---------------------------|
| 1 Probe cover ejection button | 7 Probe lens              |
| 2 Measurement indicator       | 8 Battery cover           |
| 3 Scan button                 | 9 Lug                     |
| 4 Display screen              | 10 Cradle and probe cover |
| 5 °C / °F button              | 11 Probe cover            |
| 6 Memory button               |                           |


## HINTS ON TAKING TEMPERATURES

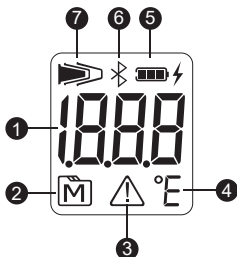1. As with other thermometers, you may observe slight variations in consecutive measurements. It is recommended that you take 3 temperature readings and use the highest one for the following situations:

- Infants younger than 3 months old.
- Children younger than 3 years old and who have a compromised immune system and the presence / absence of fever is critical.
- When you are learning to use the thermometer.

- Do not take a reading while the patient is moving and / or talking. Wait 30 minutes after any of the following situations before taking a measurement:
  - When the body site for measurement has been covered.
  - After the patient has been exercising, swimming, or taking a bath.
  - When the patient has been exposed to extreme temperature.
- To take accurate readings, the ear must be free from excess earwax build-up.

## DISPLAY SCREEN


- |                       |  |
|-----------------------|--|
| ① Temperature reading | ⑤ Battery indicator                                      |
| ② Memory mode         | ⑥ Bluetooth symbol (ri-thermo <sup>®</sup> tymPRO+ only) |
| ③ Error message       | ⑦ Probe cover indicator                                  |
| ④ Temperature unit    |  |

## REPLACING THE BATTERY

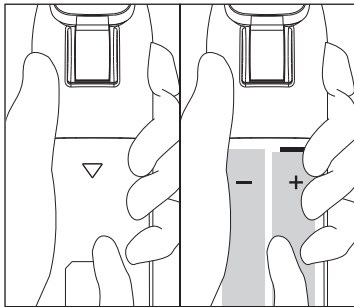The thermometer comes with two 1.5 V AA alkaline batteries. The meter will display “” to alert you when the meter power is getting low. Please follow the steps below to replace new batteries immediately.

1. Remove the battery cover. (Figure 1)
2. Remove the old batteries and replace with two 1.5V AA size alkaline batteries. (Figure 2)
3. Reattach the battery cover. Make sure the battery cover is properly closed and secured.

(Figure 1)

(Figure 2)

### NOTE:

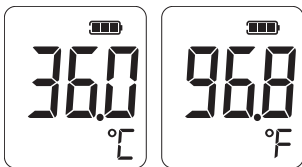
- Although the thermometer works when “” appears, we still recommend that you change the batteries to obtain an accurate result.
- Remove the batteries if stored for a long period of time.
- The batteries should be kept away out of child's reach. If they are swallowed, promptly see a doctor for help.

## CHANGING UNIT OF MEASUREMENT

This meter provides two measurement units used for indicating the body temperature, °C or °F, for your preferred selection.

STEP 1. Press the  button to turn on the thermometer first.

STEP 2. Press the °C / °F button to select. Press the  button to begin the measurement or keep the meter in idle for 60 seconds to automatically turn it off.

## RESTRICTIONS OF USE

This thermometer is clinically proven to produce accurate temperature measurements. However, please be advised if the patient has the following situations:

- The accuracy cannot be ensured for a person who has deformity in the ear such that the thermometer probe cannot be properly inserted into the ear canal.
- The accuracy cannot be ensured when blood or drainage is found in the ear canal.
- If ear drops or medications have been placed in an ear, take measurements from the other ear.
- For a person who wears ear plug or hearing aid, remove the device and wait for 15 minutes before taking temperature.

### NOTE:

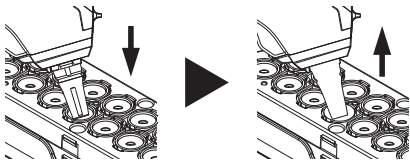
Never try to clean inside the ears. You may accidentally damage the eardrum or its surrounding tissues. Remove excess earwax only when you can reach it with a clean cloth.

## USING THE DEVICE

### Replacing Probe Cover

STEP 1. Firmly insert the thermometer probe into new probe cover in the probe cover cassette.

STEP 2. Lift the probe out of the cassette with a clean probe cover attached. Do not touch the cover. Now, you can measure the temperature.

**NOTE:**

- The used probe cover should be removed after each measurement to ensure accuracy reading.
- To ensure proper hygiene, attach a new probe cover each time and do not touch its tip.

**Measuring Temperature**

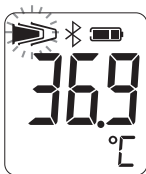
STEP 1. Press and release the  button. With a flashing “”, attach a probe cover; or, attach a probe cover to turn on the thermometer. When ready, the thermometer displays the last measurement.

STEP 2. Gently fit the probe with clean probe cover into the ear canal.


STEP3. Press and release the  button. You will hear a beep to indicate measurement is complete and thermometer may be removed. The indicator blinks during the measurement and turns solid blue when the measurement is completed.

STEP 4. Read the result. The Bluetooth symbol is for ri-thermo<sup>®</sup> tymPRO+ only.

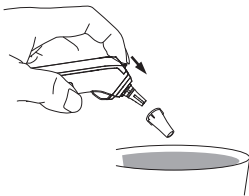STEP 5. Press ejection button to discard used probe cover into trash.

#### NOTE:

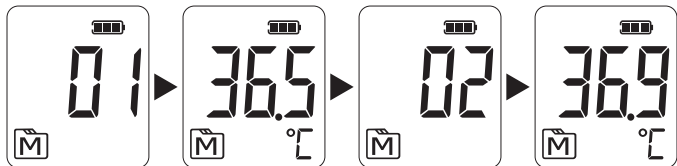
- You can turn off the thermometer by pressing and holding the  button. Or, it will automatically turn off when left idle for 1 minute.
- If the reading is  $\leq 31.9^{\circ}\text{C}$  ( $89.5^{\circ}\text{F}$ ), the display will show “Lo” with red backlight.
- If the reading is  $\geq 32^{\circ}\text{C}$  ( $89.6^{\circ}\text{F}$ ) and  $< 37.9^{\circ}\text{C}$  ( $100.3^{\circ}\text{F}$ ), the display will show the reading with green backlight.
- If the reading is  $\geq 38^{\circ}\text{C}$  ( $100.4^{\circ}\text{F}$ ) and  $< 43^{\circ}\text{C}$  ( $109.4^{\circ}\text{F}$ ), the display will show the reading with red backlight.
- If the reading is  $\geq 43.1^{\circ}\text{C}$  ( $109.5^{\circ}\text{F}$ ), the display will show “Hi” with red backlight.

## RECALLING PAST READINGS

Your thermometer stores 8 (ri-thermo<sup>®</sup> tymPRO+) / 10 (ri-thermo<sup>®</sup> tymPRO) most recent readings.

**STEP 1. Press and release the  button to turn on the thermometer.**

**STEP 2. Press the  button**, a result will be displayed after a number shown from 01 to 08 (ri-thermo<sup>®</sup> tymPRO+) / 10 (ri-thermo<sup>®</sup> tymPRO) in order of dates (latest result shows first), together with blinking "". When the memory is full, the oldest result is deleted as the new one added. When the last record is displayed in the LCD, press  button again to return to the first record.

### STEP 3. Exit the memory.

Press the  button to exit the memory mode. You can turn off the thermometer by pressing and holding the  button. Or, it will automatically turn off when left idle for 1 minute.

## BLUETOOTH PAIRING (ri-thermo<sup>®</sup> tymPRO+ ONLY)

You can transmit your data from the meter to your compatible devices (e.g. RVS-100 with USB BT Dongle) via Bluetooth. Please note that you must complete the pairing between the meter and Bluetooth receiver before transmitting data.

### Pairing with your device

1. Turn on the Bluetooth function on your device.
2. Two methods are provided to activate the Bluetooth function on the device:
  - After finishing each measurement, the Bluetooth symbol will appear (Do not eject probe cover during pairing/transfer process).
  - Turn on the meter, press and hold the Scan  and  buttons simultaneously until "CLr" appears on the display. It means that the Bluetooth of the meter is turned on automatically.
3. Follow the instruction on your device to pair the meter (e.g. search for the meter and then connect it).

4. After successfully pairing the meter with your device, the Bluetooth function of the meter will be activated automatically after each measurement. The meter is ready for transmitting the data to your device.

**NOTE:**

- While the meter is in transmission mode, it will be unable to perform a test.
- Make sure your device supports Bluetooth Smart Technology. Also make sure the Bluetooth setting on your device is turned on and the meter is within the receiving range before transmitting the data.

## **ABOUT NORMAL BODY TEMPERATURE & FEVER**

Body temperature can vary from one individual / person to next. It also varies by location on the body and time of day. Below shows the statistical normal ranges from different sites. Please keep in mind that temperatures measured from different sites, even at the same time, should not be directly compared.

Fever indicates that the body temperature is higher than normal. This symptom may be caused by infection, overdressing or immunization. Some people may not experience fever even when they are ill. These include, but are not limited to, infants younger than 3 months old, persons with compromised immune systems, persons taking antibiotics, steroids or antipyretics (aspirin, ibuprofen, acetaminophen), or persons with certain chronic illnesses.

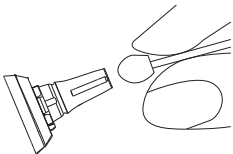
### **Body Site Normal Temperature Range\*<sup>1</sup>**

<b>Oral</b>	<b>normal oral temperature is 37°C (98.6°F)</b>
Rectal / Ear	0.3°C to 0.6°C (0.5°F to 1°F) higher than oral temperature
Axillary / Forehead	0.3°C to 0.6°C (0.5°F to 1°F) lower than oral temperature

\*1. <https://wa.kaiserpermanente.org/kbase/topic.jhtml?docId=tw9223>

## MAINTENANCE

- The probe is not waterproof. Please wipe with a clean and dry cotton swab to clean the probe on the inside.

- Do not attach new probe cover for 30 minutes. The cleaning process cools the probe and can affect accuracy.
- The body of the thermometer is not water-resistant. Never put the thermometer under a running tap or submerge it into water. Use a soft and dry cloth to clean it. Do not use abrasive cleaners.
- Store the thermometer in a cool and dry location. Free from dust and away from direct sunlight.

# TROUBLESHOOTING

The table below shows problems you may encounter. All error messages below would be shown together with backlight. Please follow “what to do” to resolve problems. If the problem still exists, please call your local dealer for help.

MESSAGE	WHAT IT MEANS	WHAT TO DO
	Appear when environmental temperature is below / above system operation range.	Put the thermometer under operating temperature range of 10°C to 40°C (50°F to 104°F).
	Problem with the thermometer.	Read the operation instruction and start a new measurement. If the problem persists, contact the customer service.
	Battery is low.	Please replace batteries as soon as possible.
	Problems with the temperature measurement for the object.	Review the instructions and re-start the measurement procedure.
	The temperature reading is below 32°C (89.6°F).	Please follow this instruction to take a measurement again.
	The temperature reading is above 43°C (109.4°F).	

# SYMBOL INFORMATION

SYMBOL	REFERENT	SYMBOL	REFERENT
	Consult instructions for use		Type BF applied part
	Manufacturer		Temperature limit
	Serial number		Humidity limitation
	Caution	<b>IP22</b>	Resistant to liquid ingress
	CE mark		RoHS compliance
	Authorized representative in the European Community		
	This device does not belong to household waste and must be returned to a collection point for recycling electric and electronic devices according to local laws. If it contains batteries, the batteries should be removed and disposed in accordance with locations for separate collection of spent batteries.		

# SPECIFICATIONS

Model No.: TD-1107

Dimension: 149.56mm (L) x 41.7mm (W) x 56.15mm (H)

Weight (exclude batteries): 80.4 (tymPRO+) / 79.6g (tymPRO)

Battery: 2 x 1.5 V AA alkaline batteries

Battery Life: 3000 times

Displayed Temperature Range: 32°C to 43°C (89.6°F to 109.4°F)

Display Resolution: 0.1°C / 0.1°F

Accuracy: Meet the accuracy requirement specified in ASTM E1965-98

- $\pm 0.2^{\circ}\text{C}$  ( $\pm 0.4^{\circ}\text{F}$ ) for the range of  $35^{\circ}\text{C}$  to  $42^{\circ}\text{C}$  ( $95^{\circ}\text{F}$  to  $107.6^{\circ}\text{F}$ )
- $\pm 0.3^{\circ}\text{C}$  ( $\pm 0.5^{\circ}\text{F}$ ) for the range of  $<35.0^{\circ}\text{C}$  ( $95^{\circ}\text{F}$ ) or  $>42.0^{\circ}\text{C}$  ( $107.6^{\circ}\text{F}$ ).

Temperature Unit:  $^{\circ}\text{C}$  (Default) or  $^{\circ}\text{F}$

System Operating Conditions:  $10^{\circ}\text{C}$  to  $40^{\circ}\text{C}$  ( $50^{\circ}\text{F}$  to  $104^{\circ}\text{F}$ ), 30% to 85% RH (non-condensing), 700 hPa to 1060 hPa

Storage / Transport Conditions:  $-20^{\circ}\text{C}$  to  $60^{\circ}\text{C}$  ( $-4^{\circ}\text{F}$  to  $140^{\circ}\text{F}$ ), 30% to 85% RH (non-condensing)

Memory Capacity: 8 (tymPRO+) / 10 (tymPRO) measurements

External Output (ri-thermo<sup>®</sup> tymPRO+ only): Bluetooth (Frequency: 2.45GHz; Bandwidth: 170MHZ; Modulation: GFSK; ERP: 5.46 dBi)

Expected Service Life: 3 years

IP Classification: IP22

The specifications may be changed without prior notice.

## REFERENCE STANDARDS

### Device Standard:

Device Corresponds to the requirements of the standard for infrared thermometers. ASTM E1965-98, EN ISO 80601-2-56:2017, EN 60601-1-2:2015, EN 60601-1:2006+A12:2014, EN 60601-1-6:2010.

### Electromagnetic Compatibility:

Device fulfills the stipulations of the standard EN 60601-1-2.

The stipulations of EU-Directive 93/42/EEC for Medical Devices Class IIa have been fulfilled.

**Manufacturer's declaration-electromagnetic emissions**

The device is intended for use in the electromagnetic environment specified below.  
The customer or the user of the device should assure that it is used in such an environment.

<b>Emission test</b>	<b>Compliance</b>	<b>Electromagnetic environment-guidance</b>
RF emissions CISPR 11	Group 1	The device uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The device is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Not applicable	
Voltage fluctuations / flicker emissions IEC 61000-3-3	Not applicable	

**Manufacturer's declaration-electromagnetic immunity**

The device is intended for use in the electromagnetic environment specified below.  
The customer or the user of the device should assure that it is used in such an environment

<b>Immunity test</b>	<b>IEC 60601 test level</b>	<b>Compliance level</b>	<b>Electromagnetic environment-guidance</b>
Electrostatic discharge(ESD) IEC 61000-4-2	Contact:±8 kV Air:±2 kV,±4 kV,±8 kV,±15 kV	Contact:±8 kV Air:±2 kV,±4 kV,±8 kV,±15 kV	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%
Electrical fast transient/burst IEC 61000-4-4	± 2kV for power supply lines ± 1kV for input/output lines	Not applicable Not applicable	Mains power quality should be that of a typical home and professional environment.
Surge IEC 61000-4-5	± 0.5kV, ±1kV line(s) to line(s) ± 0.5kV, ±1kV,±2kV line(s) to earth	Not applicable Not applicable	Mains power quality should be that of a typical home and professional environment.
Voltage Dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	Voltage dips: 0 % UT; 0,5 cycle 0 % UT; 1 cycle 70 % UT; 25/30 cycles  Voltage interruptions: 0 % UT; 250/300 cycle	Voltage dips: Not applicable Not applicable Not applicable  Voltage interruptions: Not applicable	Mains power quality should be that of a typical home and professional environment. If the user of the device requires continued operation during power mains interruptions, it is recommended that the device be powered from an uninterruptible power supply or a battery.
Power frequency(50, 60 Hz) magnetic field IEC 61000-4-8 U	30 A/m 50 Hz or 60 Hz	30 A/m 50 Hz and 60 Hz	The device power frequency magnetic fields should be at levels characteristic of a typical location in a typical home and professional healthcare environment.

NOTE UT is the a.c. mains voltage prior to application of the test level.

**Manufacturer's declaration-electromagnetic immunity**

The device is intended for use in the electromagnetic environment specified below.  
The customer or the user of the device should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment-guidance
Conducted RF IEC 61000-4-6	3 Vrms: 0,15 MHz – 80 MHz 6 Vrms: in ISM and amateur radio bands between 0,15 MHz and 80 MHz  80 % AM at 1 kHz	Not applicable Not applicable	Portable and mobile RF communications equipment should be used no closer to any part of the device including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.
Radiated RF IEC 61000-4-3	10 V/m 80 MHz – 2,7 GHz 80 % AM at 1 kHz	10 V/m 80 MHz – 2,7 GHz 80 % AM at 1 kHz	Recommended separation distance: $d = 1,2 \sqrt{P}$ $d = 1,2 \sqrt{P}$ 80MHz to 800 MHz $d = 2,3 \sqrt{P}$ 800MHz to 2,7 GHz Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m). Interference may occur in the vicinity of equipment marked with the following symbol: 

NOTE1 At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the device is used exceeds the applicable RF compliance level above, the device should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the device.

b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

**Recommended separation distance between  
portable and mobile RF communications equipment and the device**

The device is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the device can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the device as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = 1,2 \sqrt{P}$	150 kHz to 80 MHz $d = 1,2 \sqrt{P}$	150 kHz to 80 MHz $d = 1,2 \sqrt{P}$
0,01	N/A	N/A	N/A
0,1	N/A	N/A	N/A
1	N/A	N/A	N/A
10	N/A	N/A	N/A
100	N/A	N/A	N/A

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where p is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

**Manufacturer's declaration-electromagnetic immunity  
Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communications equipment**

The device is intended for use in the electromagnetic environment specified below.  
The customer or the user of the device should assure that it is used in such an environment.

Test frequency (MHz)	Band <sup>a)</sup> (MHz)	Service <sup>a)</sup>	Modulation <sup>b)</sup>	Maximum power (W)	Distance (m)	IMMUNITY TEST LEVEL (V/m)	Compliance LEVEL (V/m)
385	380 – 390	TETRA 400	Pulse modulation <sup>b)</sup> 18 Hz	1,8	0,3	27	27
450	430 – 470	GMRS 460, FRS 460	FM <sup>c)</sup> ±5 kHz deviation 1 kHz sine	2	0,3	28	28
710	704 – 787	LTE Band 13, 17	Pulse modulation <sup>b)</sup> 217 Hz	0,2	0,3	9	9
745							
780							
810	800 – 960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation <sup>b)</sup> 18 Hz	2	0,3	28	28
870							
930							
1 720	1 700 – 1 990	GSM 1800; CDMA 1900; GSM 1900; DECT, LTE Band 1, 3, 4, 25; UMTS	Pulse modulation <sup>b)</sup> 217 Hz	2	0,3	28	28
1 845							
1 970							
2 450	2 400 – 2 570	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulation <sup>b)</sup> 217 Hz	2	0,3	28	28
5 240	5 100 – 5 800	WLAN 802.11 a/ n	Pulse modulation <sup>b)</sup> 217 Hz	0,2	0,3	9	9
5 500							
5 785							

NOTE: If necessary to achieve the IMMUNITY TEST LEVEL, the distance between the transmitting antenna and the ME EQUIPMENT or ME SYSTEM may be reduced to 1 m. The 1 m test distance is permitted by IEC 61000-4-3.

a) For some services, only the uplink frequencies are included.

b) The carrier shall be modulated using a 50 % duty cycle square wave signal.

c) As an alternative to FM modulation, 50 % pulse modulation at 18 Hz may be used because while it does not represent actual modulation, it would be worst case.

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**Rudolf Riester GmbH**
**TaiDoc Technology Corporation**
**MedNet EC-REP GmbH**