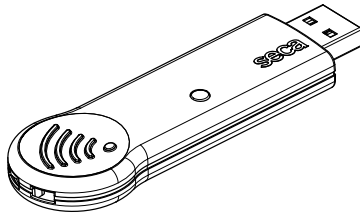
# seca 456

<b>D</b>	<b>Bedienungsanleitung und Garantieerklärung</b> . . . . .	<b>4</b>
<b>GB</b>	<b>Instruction manual and guarantee</b> . . . . .	<b>6</b>
<b>F</b>	<b>Mode d'emploi et garantie</b> . . . . .	<b>8</b>
<b>I</b>	<b>Manuale di istruzioni e garanzia</b> . . . . .	<b>10</b>
<b>E</b>	<b>Manual de instrucciones y garantía</b> . . . . .	<b>12</b>
<b>DK</b>	<b>Betjeningsvejledning og garantibevis</b> . . . . .	<b>14</b>
<b>S</b>	<b>Bruksanvisning och garanti</b> . . . . .	<b>16</b>
<b>N</b>	<b>Bruksanvisning og garantierklæring</b> . . . . .	<b>18</b>
<b>FIN</b>	<b>Käyttöohje ja takuu</b> . . . . .	<b>20</b>
<b>NL</b>	<b>Bedieningshandleiding en garantieverklaring</b> . . . . .	<b>22</b>
<b>P</b>	<b>Instruções de utilização e declaração de garantia</b> . . . . .	<b>24</b>
<b>GR</b>	<b>Οδηγίες χειρισμού και εγγύηση</b> . . . . .	<b>26</b>
<b>PL</b>	<b>Instrukcja obsługi i gwarancja</b> . . . . .	<b>28</b>
<b>RU</b>	<b>Инструкция по эксплуатации и гарантийное свидетельство</b> . . . . .	<b>30</b>
<b>J</b>	<b>使用説明書および保証</b> . . . . .	<b>32</b>
<b>CN</b>	<b>说明书和保证</b> . . . . .	<b>34</b>

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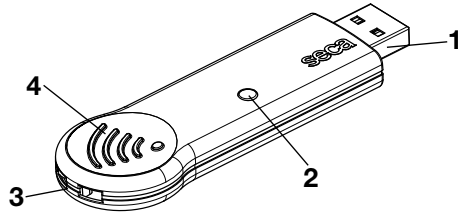
## 1. INTENDED USE

The seca **seca 360° wireless USB adapter 456** is an accessory for operating a PC in a **seca 360° wireless** network. The USB adapter allows wireless data transmission between the PC and other **seca 360° wireless** devices. The USB adapter only works on PCs with seca PC software installed.

## 2. SAFETY INFORMATION

- Follow the instructions in this user manual.
- Keep the user manual in a safe place.
- Follow the instructions in your PC manual and those for the **seca 360° wireless** devices used.
- To prevent malfunctions, do not operate or store the device in an environment in which it is exposed to high humidity, vibrations, dust, heat or direct sunlight.
- Do not start up the device immediately after transporting it. Severe temperature fluctuations and changes in humidity may lead to moisture forming on the device. This may damage both the device itself and the PC to which it is connected.
- Make sure the device cannot inadvertently be disconnected from the PC; this will prevent malfunctions and use not in accordance with intended use.
- Make sure HF devices, such as mobile phones and televisions, are kept at a minimum distance of approx. 1 metre to prevent incorrect measurements or interference with wireless transmission.
- Do not open the device housing. This may lead to damage to the device. No warranty will be provided by seca in such cases.

### 3. OVERVIEW

No.	Element/symbol	Function/meaning
1	USB stick	Plugged into a free USB port on the PC
2	LED	<ul style="list-style-type: none"> <li>• Red:               <ul style="list-style-type: none"> <li>- USB adapter correctly detected by PC</li> </ul> </li> <li>• Blue:               <ul style="list-style-type: none"> <li>- data being transmitted</li> </ul> </li> </ul>
3	Security loop	For attaching a lanyard, for example: facility to prevent stick being removed from the USB port inadvertently
4		Device is equipped with a radio module, data are received and transmitted wirelessly

### 4. OPERATION

The USB adapter is supplied ready to use.

1. Plug the USB adapter into a free USB port on your PC.
2. Make sure the USB adapter cannot be removed from the USB port inadvertently.

**NOTE**

Information about operating the USB adapter in a **seca 360° wireless** network can be found in the administrator manual for the seca PC software in use.

### 5. CLEANING

No cleaning is required in normal use.

- ◆ If required, clean the housing with a damp cloth.

**ATTENTION!**

**Damage to device and malfunctions may be caused by unsuitable cleaning agents**

- Do not use solvents, corrosive liquids or spray cleaners.

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FCC ID: X6T182A01

IC: 8898A-182A01

This device complies with Part 15 of the FCC Rules and with RSS-210 of Industry Canada. Operation is subject to the following two conditions. (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

## **NOTE:**

This device complies with Part 15 of the FCC Rules and with RSS-210 of Industry Canada. Operation is subject to the following two conditions:

- This device may not cause harmful interference.
- This device must accept any interference received, including interference that may cause undesired operation.

## **NOTE:**

Changes or modifications made to this equipment not expressly approved by seca may void the FCC authorization to operate this equipment.

## **NOTE:**

Radiofrequency radiation exposure Information: This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. This equipment should be installed and operated with minimum distance of 1 m between the radiator and your body. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.