

RAPID DIAGNOSTICS

osom[®]

iFOB Test

FOR THE DETECTION OF FECAL OCCULT BLOOD/HUMAN HEMOGLOBIN



www.osomtests.com



FEATURES:

- ✓ **ACCURATE;** Specific to human hemoglobin
- ✓ **FAST;** Results in 5-10 minutes
- ✓ **SAMPLE;** Easy three step procedure with <1 minute hands on time
- ✓ **IMPROVE PATIENT COMPLIANCE;** No dietary or drug restrictions
- ✓ **CUSTOM CARE;** Technical support staffed with certified medical technologists




OSOM[®]

iFOB Test


Sekisui Diagnostics, LLC (AMERICAS)
 4 Hartwell Place Lexington, MA 02421
 Phone: 800 332 1042
 Fax: 800 762 6311
 Email: osomrapidtests@sekisui-dx.com
 ISO 13485 Certified

Sekisui Diagnostics (UK) Limited (INTERNATIONAL)
 Liphook Way, Allington, Maidstone, Kent, ME16 0LQ, UK
 Phone: +44 1622 607800 Fax: +44 1622 607801
 Email: info@sekisui-dx.com
 ISO 9001 Certified, ISO 13485 Certified


TEST PROCEDURE / RESULTS

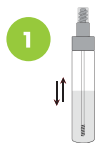

Shake tube vigorously for 20-30 seconds to ensure sample is well mixed.

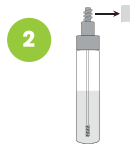


Unscrew tip cap on the stool collection tube.

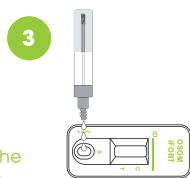

Squeeze tube gently to dispense three (3) drops into the sample well.




Read the result in 5-10 MINUTES

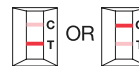

(examples of positive results)
Positive



(example of negative result)
Negative

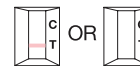

(examples of invalid results)
Invalid

DESCRIPTION	CONFIGURATION	CATALOG #	STORAGE	CPT CODE
OSOM IFOB TEST	25 TESTS	1002	ROOM TEMP	DIAGNOSTIC 82274QW SCREENING G0328QW
OSOM IFOB POSITIVE & NEGATIVE CONTROLS	2 VIALS (1 POS; 1 NEG)	1000	ROOM TEMP	

*Additional Accessories (Patient Mailers and Stool Collection Tubes) are available.
 OSOM[®] is a registered U.S. trademark of Sekisui Diagnostics, LLC. © 2015 Sekisui Diagnostics, LLC. All rights reserved.

