

RAPID DIAGNOSTICS


osom[®]

BVBLUE[®] Test

FOR THE DETECTION OF SIALIDASE ACTIVITY FROM GARDNERELLA, BACTEROIDES, PREVOTELLA, & MOBILUNCUS



COMPARISON OF BV
DETECTION METHODS*

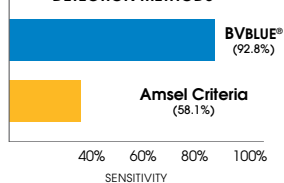

www.osomtests.com



FEATURES:

- ✓ **ACCURATE;** 92.8% sensitive, 98% specific vs. gram stain
- ✓ **FAST;** Less than 1 minute hands-on time; result in 10 minutes
- ✓ **SAMPLE;** Vaginal swab
- ✓ **OBJECTIVE;** Instant color change provides clear, easy-to-read results
- ✓ **TRAINING;** Online training and certification; www.osomtraining.com
- ✓ **CUSTOM CARE;** Technical support staffed with certified medical technologists



OSOM[®]

BVBLUE[®] Test

Sekisui Diagnostics, LLC (AMERICAS)
 4 Hartwell Place Lexington, MA 02421
 Phone: 800 332 1042
 Fax: 800 762 6311
 Email: osomrapidtests@sekisui-dx.com
 ISO 13485 Certified

Sekisui Diagnostics (UK) Limited (INTERNATIONAL)
 Liphook Way, Allington, Maidstone, Kent ME16 0LQ, UK
 Phone: +44 1622 607800 Fax: +44 1622 607801
 Email: info@sekisui-dx.com
 ISO 9001 Certified, ISO 13485 Certified

TEST PROCEDURE / RESULTS


Insert swab into pre-filled vial and mix.



Let stand for **10 MINUTES** at room temperature.




Add 1 drop of Developer Solution to vial and mix.

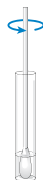

Read results.

POSITIVE

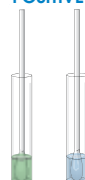

NEGATIVE



DESCRIPTION

CONFIGURATION

CATALOG #

STORAGE

CPT CODE

OSOM[®] BVBLUE[®] TEST

OSOM[®] BVBLUE[®] CONTROL KIT

25 TESTS

183

REFRIGERATED

87905QW

184

REFRIGERATED

OSOM[®] VAGINITIS TRANSPORT SYSTEM (TRICH & BV)

OSOM[®] BVBLUE[®] TRANSPORT SWAB

50 SWAB SETS

30 SWABS

185

186

ROOM TEMP

ROOM TEMP

