

Clinistix UTI Test Strips

clinistix.siemens-healthineers.com/us



A simple urinary tract infection (UTI) home test can be a fast, accurate, and reliable method for screening a UTI. Supported by nearly 80 years of innovation and leadership in the field of urinalysis, the new Siemens Healthineers Clinistix portfolio of at home urinalysis tests use the same chemistry as those strips found in doctors offices.

Overview

- Clinistix™ UTI Test Strips use the same dry reagent chemistry as the market-leading, professional-use Multistix® Reagent Strips for urinalysis.
- Self-guided test strips are used to detect nitrite and leukocytes (white blood cells, or WBC) in urine samples.
- Combining the results of nitrite and WBC is a better predictor of the presence or absence of urinary tract infection than testing for nitrite alone.

Specifications

- Visually interpreted, no analyzer required; qualitative results for nitrite and semiquantitative results for WBC.
- Results in 2 minutes.
- Room-temperature storage.
- 18-month shelf life (from date of manufacture).
- Urine sample can be collected mid-stream or dipped.
- FDA 510(k)-cleared: K960888.
- Each strip consists of a 3.25-inch white polystyrene strip (0.4 inches wide) with two reagent pads laminated to the strip.

Shipper and Pallet Configuration

Quantity per case:	24
Shipper internal dimensions:	267 x 181 x 109 mm
Cases per pallet:	198
Pallet dimensions:	1100 x 1040 x 1203 mm

Ordering Information

SMN:	11643375
UPC code:	630414631677
Product name:	Clinistix UTI Test Strips
Number of tests/package:	5 strips
Box dimensions:	43 x 105.5 x 43 mm

Clinistix UTI Test Strips: How do they work?


1 Apply urine to the strip.

Urinate directly onto the strip.
Begin urinating. After some urine has passed, hold the test pads in the urine stream for 1–2 seconds.

OR

Urinate into a clean cup and dip the strip into the sample.

Urinate into a clean, dry paper or plastic cup. Dip the test strip into the urine.

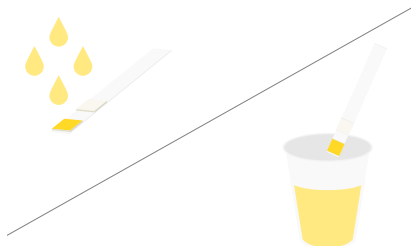


2 Wait.

Start timer. Lay the test strip, pads up, on a folded tissue.




Leave the test strip on the folded tissue for 2 minutes.

3 Read results.

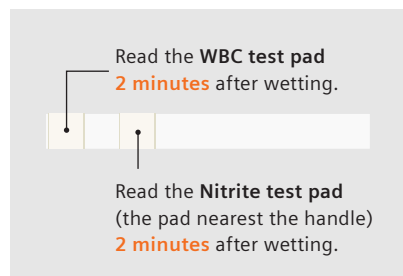

After waiting for 2 minutes, match the color of each test pad to the color chart for that test on the test strip bottle.

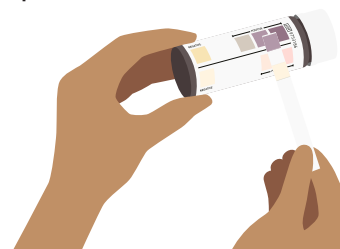

Record Nitrite and WBC results where indicated on the package insert.

Date of Test	/2022/05/17/					
WBC (2 min.)	N					
Nitrite (2 min.)	P					
(Example)		N = Negative		P = Positive		

Clinistix, Multistix, and all associated marks are trademarks owned or licensed by Siemens Healthcare Diagnostics Inc., or its affiliates. All other trademarks and brands are the property of their respective owners.

Product availability may vary from country to country and is subject to varying regulatory requirements. Please contact your local representative for availability.

Always consult with your doctor to confirm the diagnosis of a Urinary Tract Infection and advise on treatment.

Siemens Healthineers Headquarters

Siemens Healthcare GmbH
Henkestr. 127
91052 Erlangen, Germany
Phone: +49 9131 84-0
siemens-healthineers.com

Legal Manufacturer

Siemens Healthcare Diagnostics Inc.
Laboratory Diagnostics
511 Benedict Avenue
Tarrytown, NY 10591-5005
USA
Phone: +1 914-631-8000

Local Contact Information

Siemens Healthcare Diagnostics Inc.
Point of Care Diagnostics
2 Edgewater Drive
Norwood, MA 02062-4637
USA
Phone: +1 781-269-3000