



Vista Non-Powered Stand Aid

Model No. 51871

Operator's Manual

51928 R05 / AN 240401

April 2024

WARRANTY

Wy'East Medical Corporation (Wy'East) warrants to the original user the Vista Non-Powered Stand Aid manufactured by the company is free from defects in material and workmanship under normal use and service. The term of warranty from the date of product receipt is for a period of two (2) years on mechanical lift components. During this term Wy'East will provide to the user, at Wy'East's option, new or repaired replacement components.

In order to initiate a warranty claim, original user must notify Wy'East within the applicable warranty period specifying the nature of the claim. Wy'East will issue instructions as to the procedure to follow. Wy'East reserves the right to confirm the existence of any claimed defect and to determine whether any damage was caused by neglect, misuse, unauthorized alteration or failure to follow maintenance instructions. With respect to a claim covered by this warranty, the defective part will be replaced or repaired at no charge, provided the defective part is returned to Wy'East. The user has the responsibility for installation. Assistance is available by calling [\(503\) 657-3101](tel:5036573101) or through email at info@weeastmed.com. This is the exclusive remedy for a claim under this warranty and in no event shall Wy'East be liable for consequential damages. THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES EXPRESSED OR IMPLIED.

PLEASE

Read this entire manual before operating the Vista Non-Powered Stand Aid.

This warranty applies to the Wy'East Vista Non-Powered Stand Aid only.
For questions or information regarding the Vista Non-Powered Stand Aid, contact:

Wy'East Medical Corporation
16700 SE 120th Ave.
Clackamas, OR 97015, USA
[\(503\) 657-3101](tel:5036573101)
[\(800\) 255-3126](tel:8002553126)

weeastmedical.com
info@weeastmed.com

Contents

INTRODUCTION

HOW TO USE THIS MANUAL:	4
DEFINITIONS:	4
SAFETY CONSIDERATIONS FOR ALL PROCEDURES:	6
PRIOR TO FIRST USE:	6
FEATURES:	7
Install/Uninstall Footrest:	7
Shin Guard Cam Lever:	7
Shin Guard Adjustment Toward/Away from Patient:	7
Shin Guard Adjustment Up/Down:	7
Seat Position: Up	8
Seat Position: Out	8
Seat Position: Up and Out	8
Seat Height Adjustment:	9
Legs Open/Close Function:	9
OPERATING THE VISTA NON-POWERED STAND AID:	10

MAINTENANCE & ASSISTANCE

TECHNICAL SPECIFICATIONS:	12
PERIODIC MAINTENANCE:	13
ONCE A MONTH	13
TROUBLESHOOTING:	13
FACTORY ASSISTANCE:	14
REPLACEMENT ITEMS:	14
RETURNING ITEMS:	14
REPLACEMENT PARTS:	15
NOTES:	16
CONTACT INFORMATION	17

Consistent with our policy of continuing product improvement, WY'EAST MEDICAL CORPORATION reserves the right to change these specifications without notice or obligation.

INTRODUCTION

HOW TO USE THIS MANUAL

This manual is designed to familiarize you with the operation of the Vista Non-Powered Stand Aid. Read it thoroughly before using or servicing the equipment. Keep the manual for future reference and for new members of your staff.



DO NOT ATTEMPT TO USE EQUIPMENT WITHOUT UNDERSTANDING THIS MANUAL

To ensure safe operation, read carefully the entire manual, before installing, operating, or servicing this equipment.

If anything is not completely understood, please contact your supplier for more details. Failure to comply with warnings in this manual may result in injury.

Keep this manual with the Vista Non-Powered Stand Aid and refer to it as required. Contents of this manual are subject to change without prior notice to users.

DEFINITIONS

The words **WARNING**, **CAUTION**, and **NOTE** carry specific meanings and should be carefully reviewed.

WARNING: The personal safety of the patient or user may be involved. Disregarding this information could result in injury to the patient or user.

CAUTION: These instructions point out special procedures or precautions that must be followed to avoid damaging the Vista.

NOTE: This provides special information to make maintenance easier or important instructions more clear.

MEANING OF SYMBOLS

	Caution		Safe working load (SWL)
	Consult instructions for use (IFU)		Do not step on surface
	Important		

Your Vista Non-Powered Stand Aid is designed for assisting patients with weight bearing ability to stand and transfer.

- Bed to Chair transfer.
- Wheelchair repositioning.
- Easy access for wheelchair.
- Total adjustability for convenience and usability.
- Sturdy dependability.

The Vista has a compact footprint allowing for greater maneuverability in tight spaces.




Intended for use in acute care, long-term care, outpatient care, and home care. It is not recommended for use in a shower.



Prior to using the Vista for transfer or any other procedure, the patient must be assessed by a qualified professional to determine the suitability of using the Vista with that patient. Patients should be partial weight bearing, have upper body strength and be able to follow instructions.



With the footrest removed the Vista can be used to assist in weight bearing rehabilitation and balance training.

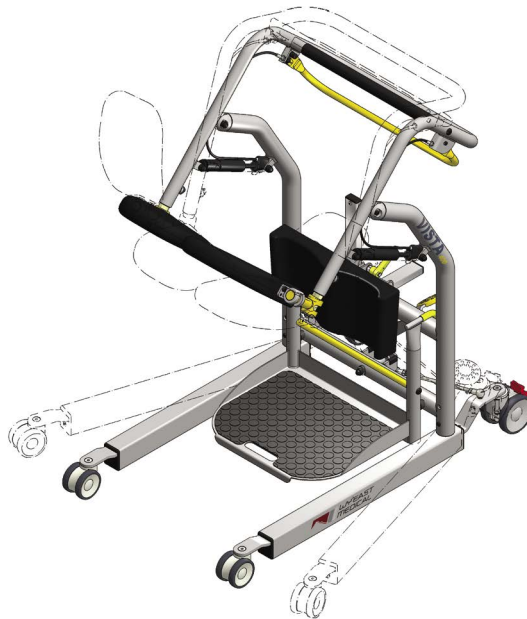



SAFETY CONSIDERATIONS FOR ALL PROCEDURES

1. The Vista is designed to be used **ONLY** on level surfaces.
2. **ALWAYS** lock the wheels while positioning the patient.
3. Prior to using the Vista for transfer, the patient must be assessed by a qualified professional to determine the suitability of using the Vista with that patient.
4. Never leave the patient unattended while in the Vista, or during any portion of the transfer procedure.
5. Ensure the patient is properly seated in the Vista before transporting. Never transport a patient while they are standing in the Vista.
6. Never exceed the safe working load. SWL is 400 lbs.
7. Do not power wash the Vista.
8. When cleaning and disinfecting, use a 5.25% chlorine bleach dilution or an EPA registered Hard Surface Disinfectant on painted and plastic surfaces. On all other surfaces use an EPA registered Hard Surface disinfectant wipe that does not contain sodium hypochlorite (bleach).
9. Warranty applies only if repairs or alterations are performed by an authorized technician.

PRIOR TO FIRST USE



If any individual parts are suspected to be damaged, do not operate the parts, but return them for inspection/service.


FEATURES



Install/Uninstall Footrest:


To remove the footrest, lift up by holding the yellow cross bar and handle (see Fig.1).

- Raise the footrest 2 inches vertically.
- Remove as shown.

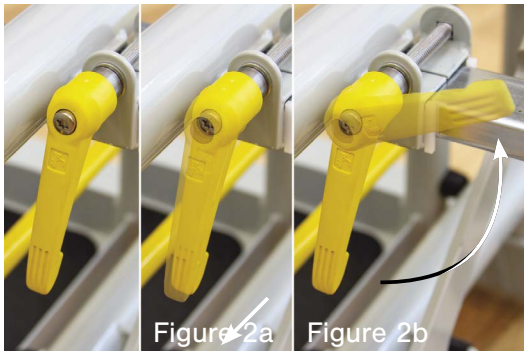

Shin Guard Cam Lever:

- To clamp, rotate the lever clockwise.
- To re-position the lever at an angle to allow clamping, pull the yellow handle back (see Fig. 2a), rotate to desired position (see Fig. 2b) and release.
- You are now ready to rotate to clamp.

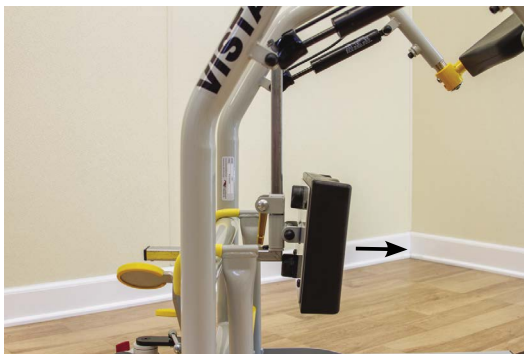

Shin Guard Adjustment Toward/Away from Patient:


Adjust shin guard to ensure that shin guard rests against the patient's shins while patient is sitting with lower leg at 90° to thigh where the thigh is parallel with the floor.



Shin Guard Adjustment Up/Down:

Adjust shin guard to ensure that the top of the shin guard is at least 2 inches below the patient's patella to avoid injury.

FEATURES

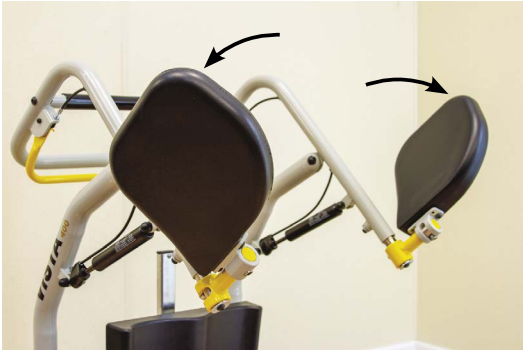

Seat Position: Up

To move seat up, pull up and rotate seat as shown.



Seat Position: Out


To move seat out, push up on yellow knuckle and rotate seat as shown.



Seat Position: Up and Out

To move seat up and out, follow the instructions for the two previous features.

FEATURES

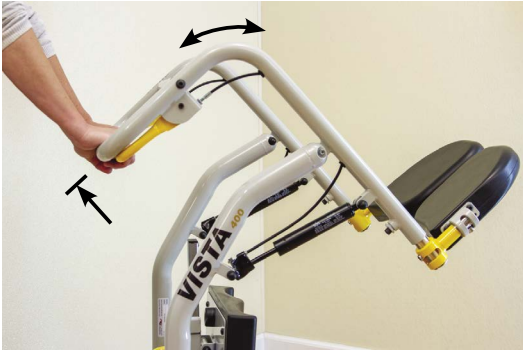

Seat Height Adjustment:

To lower or raise the seat height, squeeze the yellow lever and pull up or push down on the handlebar.

NOTE: Adding weight on the seat pads can aid in height adjustment.



Legs Open/Close Function:

1. Push down on the left yellow pedal to open legs.
2. Push down on the right yellow pedal to close legs.

Allows clearance for the caregiver to get around a chair, wheelchair or commode and properly place the feet and set the shin guard.

OPERATING THE VISTA NON-POWERED STAND AID

1. ENSURE patient is able to sit in an upright posture and the Vista is able to be positioned in front of the patient as shown (Fig. 3).
2. Move the SEATS out and OPEN the LEGS of the Vista to an appropriate width to maneuver the Vista around a chair or wheelchair (Fig. 3).
3. Move the Vista toward patient until their feet are completely supported by the footrest and their legs are in a position as close to 90 degrees as possible (Fig. 4).
4. LOCK the rear WHEELS.

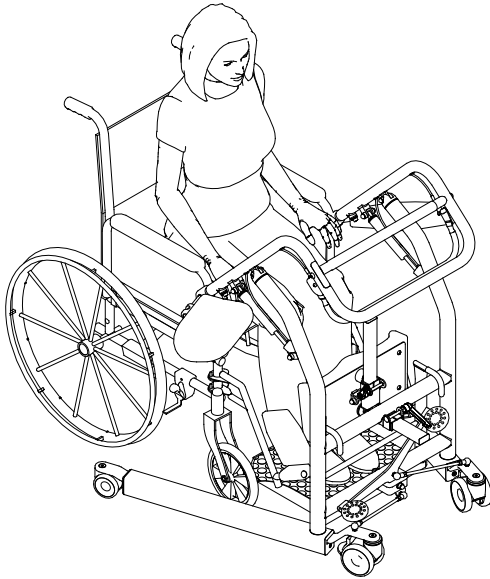


Figure 3

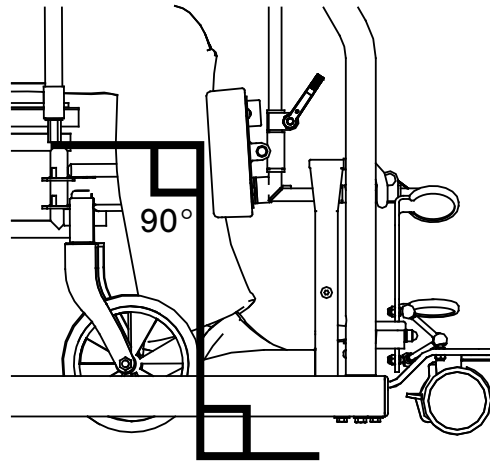


Figure 4

5. Move shin guard forward to rest against the patients shins. Lock Cam Lever.
6. Adjust the shin guard until it rests 2 or more inches below the patients knees (patellae). This prevents the patient's kneecaps being pushed upward when they stand up.
7. Instruct the patient to grab the handle bar.
8. CHECK the patients' body alignment prior to having them stand. Ensuring that the patient and the Vista are in alignment with each other will help alleviate any feelings of being off balance during the transfer.

Introduction: Operating The Vista Non-powered Stand Aid

9. Instruct the patient to stand up (Fig. 5).
10. Position the seats under the patient and have them sit (Fig. 6).
11. Adjust SEAT height by squeezing the yellow lever and moving the handlebar up or down (Fig. 7).

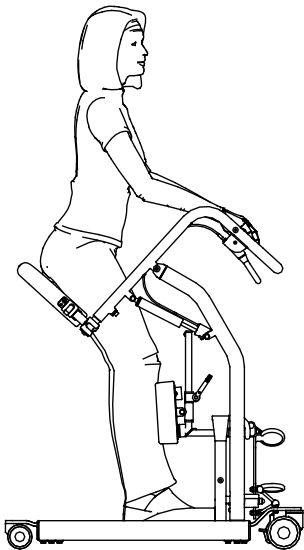


Figure 5

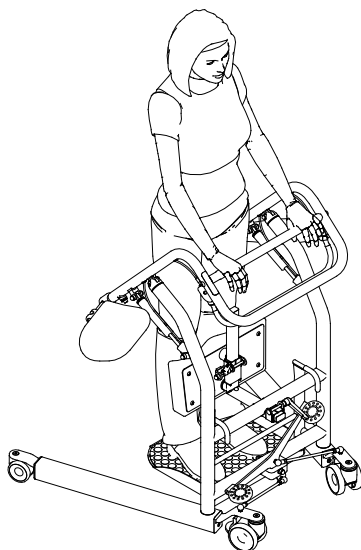


Figure 6


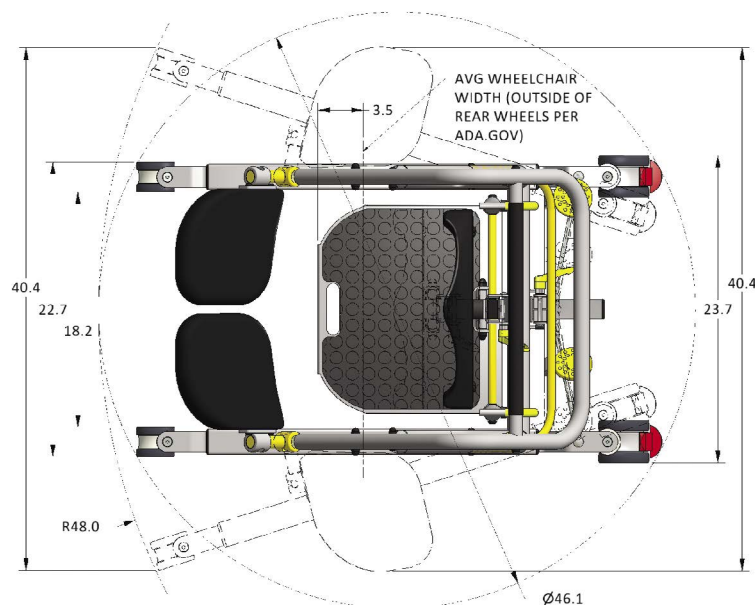


Figure 7

12. Have the patient hold the handlebar. This keeps their arms in a SAFE position for transfer.
13. DO NOT transport the patient in a standing position.
14. UNLOCK the rear casters and move the Vista to the location of intended transfer.
15. Once the patient is positioned over the intended area of seating, LOCK the rear WHEELS.
16. Have the patient stand, move the SEATS out and have the patient sit.
17. Once the patient is securely seated (ensure the patient is not seated on the edge of the destination seat!) UNLOCK rear WHEELS and move the Vista clear of the patient.
18. CLOSE LEGS.

TECHNICAL SPECIFICATIONS

PARAMETER	SPEC
• SWL 	400 lbs
• Device Part Number	51871
• Unit of Measure	1 Each
• Total Weight	92 lbs
• Weight Loose Parts (Footrest)	12 lbs
• Wheels Diameter, Front	3"
• Wheels Diameter, Rear	4"
• Turning Diameter	46.1"
• Leg Under Bed/Stretcher Clearance	3.4"
• Standard Door Width Clearance	23.7"
• Footplate Dimensions	14" x 17" x 14"
• Frame	Steel & Aluminum
• FDA Device Class	Class 1
• FDA Product Code	FSA
• FDA Regulatory Number	880.5510

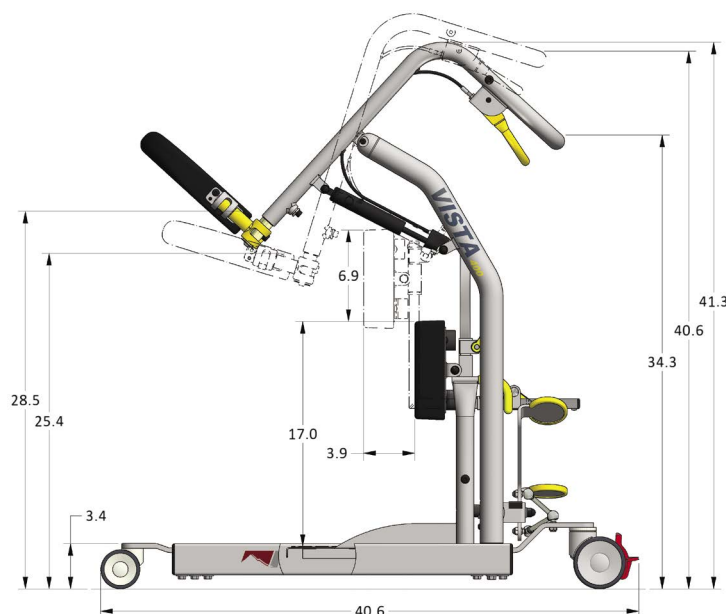

ENVIRONMENTAL CONDITIONS

	FOR SHIPMENT AND STORAGE	FOR USE
Temperature	14°F to 104°F (-10°C to 40°C)	41°F to 104°F (5°C to 40°C) ambient
Relative Humidity	20% to 80% non-condensing	20% to 80% non-condensing
Atmospheric Pressure	70 kPa to 106 kPa	70 kPa to 106 kPa

PERIODIC MAINTENANCE

The following scheduled maintenance and inspections will help to ensure trouble-free operation of your Vista Non-Powered Stand Aid.



Not recommended for use in shower.
Do not pressure-wash the unit.

ONCE A MONTH

- Check for loose or missing nuts and bolts.
- Clean unit – wipe off dust.

TROUBLESHOOTING

SYMPTOM	POSSIBLE CAUSE	ACTION
Seat height adjustment does not release or lock in position selected.	Release lever is not functioning properly.	Adjust release lever pivots.
	Gas spring release cable(s) is not functioning properly.	Adjust or replace release cable(s).
	Gas spring(s) is not functioning properly.	Replace gas spring(s).

FACTORY ASSISTANCE

Expected lifetime: 8-10 years

REPLACEMENT ITEMS:

To order replacement parts, contact Wy'East Medical Customer Service.

Include the following information with your request:

1. Serial and Model numbers of your Vista Non-Powered Stand Aid.
2. Purchase date, if available.
3. Part number and description of the replacement part needed (see "Replacement Parts" section on page 15 of manual).
4. Shipping and billing instructions, including complete address.

RETURNING ITEMS:

To return parts or goods, contact Wy'East Medical for authorization.


Contact information:

Telephone: [\(503\) 657-3101](tel:5036573101)

Email: customerservice@weastmed.com

REPLACEMENT PARTS

This manual provides maintenance and service information for the Vista Non-Powered Stand Aid. If replacement parts are required, contact WY'EAST MEDICAL with the serial number of your Vista Non-Powered Stand Aid to ensure the accuracy of replacement parts.

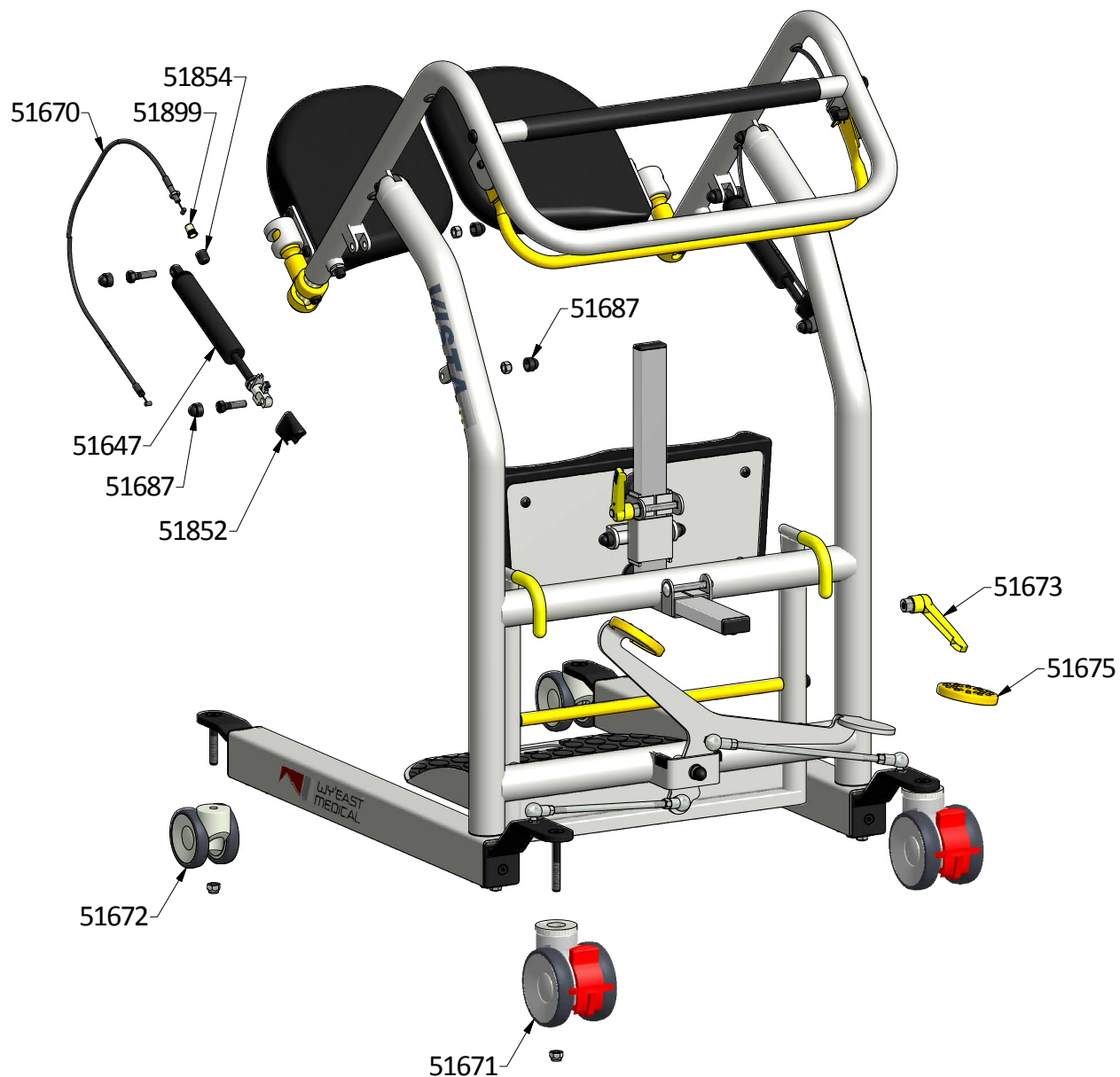

NOTES



CONTACT INFORMATION

For questions or information regarding other Wy'East Medical products, contact:

Wy'East Medical Corporation

16700 SE 120th Ave.
Clackamas, OR 97015, USA

(503) 657-3101

(800) 255-3126

wyeastmedical.com

info@wyeastmed.com

