## ENGLISH

### Symbol Descriptions

	In this document, indicates conditions or practices that could lead to illness, injury, or death ("WARNING").
	In this document, indicates conditions or practices that could damage the equipment or other property ("CAUTION").
	On the product, means consult accompanying documentation.
	Do not reuse.
	Preceeds catalog number.
	Batch code
	This end up
	Fragile
	Storage Temperature limits -4° F - 120° F (-20°C - +49°C)
	Storage Relative humidity limit 15% - 95%

### Indications for Use

#### KleenSpec Single Use Vaginal Speculum

The disposable vaginal speculum is used to dilate the vagina and expose the interior of the vagina and exterior of the cervix during pelvic examinations and other gynecological procedures. The vaginal speculum can be used with or without the illuminator.

#### KleenSpec 790 Series Cordless Illuminator

When used with the vaginal speculum, the cordless illuminator provides illumination during pelvic examinations and other gynecological procedures, such as pap smears, dilation and curettage (D&C), biopsy, and electroscopy. The illuminator can be used independent of the speculum as a general-purpose light source.

### Warnings and Cautions

**WARNING:** Inspect each KleenSpec® box for shipping damage.

**WARNING:** Inspect speculum prior to use. Do not use if damaged.

**WARNING:** If a speculum is damaged, do not use any specula from the same case.

**WARNING:** Specula, when in transit or storage, may be subject to damage beyond the control of the manufacturer or supplier.

**WARNING:** Never use the speculum with laser equipment.

**WARNING:** Treat used vaginal specula as bio hazardous infectious material. Dispose of used vaginal specula in suitable disposal unit or in accordance with local regulations.

**WARNING:** The speculum is not compatible with gamma irradiation or autoclave sterilization.

**CAUTION:** Used by trained personnel only.

**CAUTION:** Rx Only. U.S. federal law restricts this device to sale by or on the order of a physician.

### Warranty

Welch Allyn warrants the speculum to meet Welch Allyn’s specifications for the Product at the time of purchase and to be free from original defects in material and workmanship under normal use and service. Specula are intended for single use only.

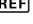
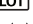
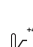
Welch Allyn should be notified via telephone of any defective product and the item should be immediately returned, securely packaged and postage prepaid to Welch Allyn. Loss or damage in shipment shall be at the purchaser’s risk.

Welch Allyn will not be responsible for loss associated with the use of any Welch Allyn product that has been altered.

**THIS WARRANTY IS EXCLUSIVE AND IN LIEU OF ANY IMPLIED WARRANTY OR MERCHANTABILITY, FITNESS FOR PARTICULAR PURPOSE, OR OTHER WARRANTY OF QUALITY, WHETHER EXPRESSED OR IMPLIED. WELCH ALLYN WILL NOT BE LIABLE FOR ANY INCIDENTAL OR CONSEQUENTIAL DAMAGES.**

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