



# 2572/2573 Blood Drawing Chair Parts List

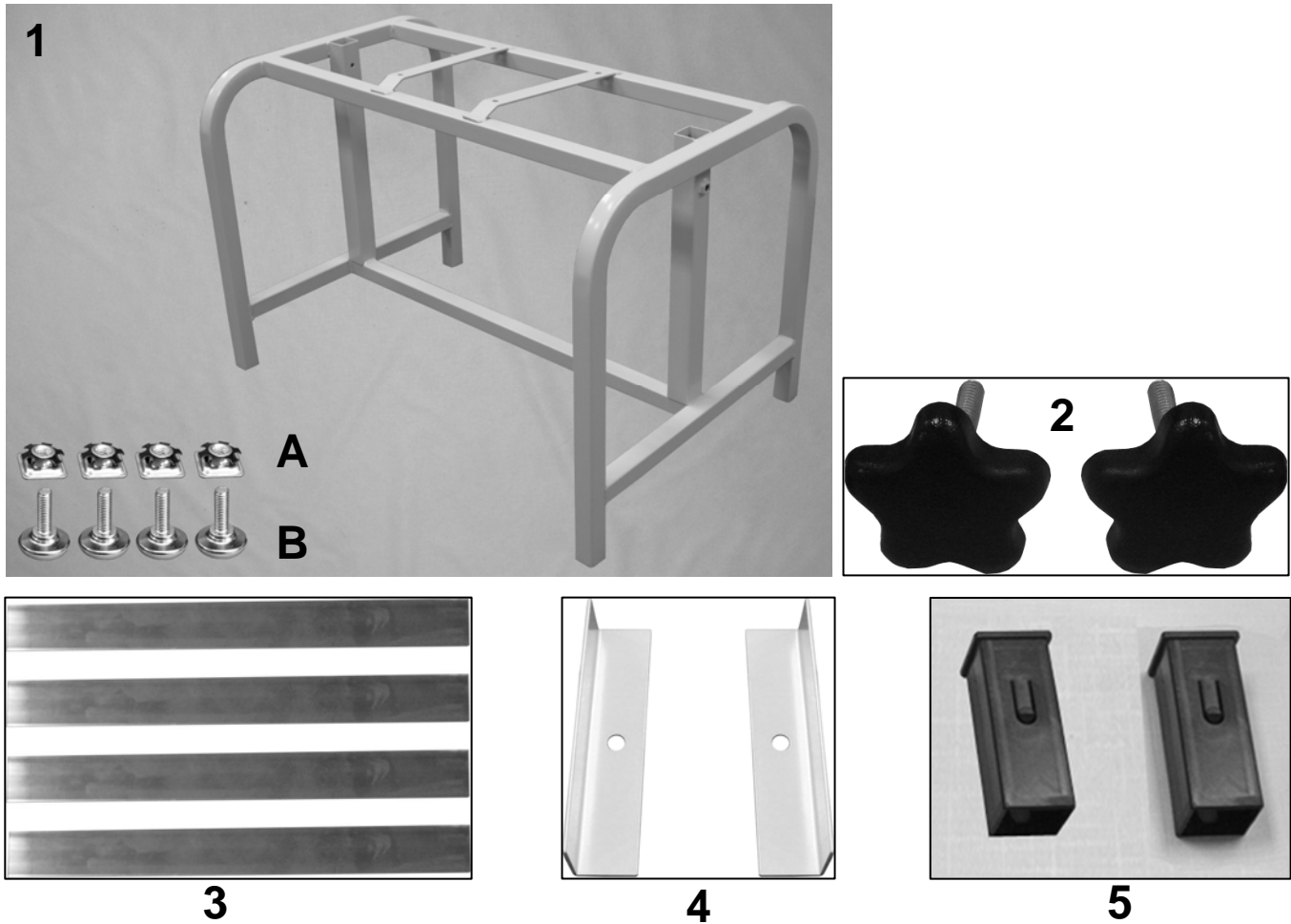

PARTS LIST				
Image Number	Note	Part Number	Description	QTY per Product
1	A,B,F	226101	WELDED MAIN FRAME (570-573 <b>B</b> STYLE)	1
1A		501060	5/16" METAL GLIDE INSERT 570/571	4
1B		500350	GLIDE NON-SWIVEL ZINC 5/16X1" NP LN	4
2	C, D, E	501210	STAR KNOB, BLK 1/4-20 X 1 1/2" STL STUD	1*
2	F	501215	STAR KNOB, BLK 5/16-18 X 1 1/2" STL STUD	1*
3	C	226250	570 ARM COLUMN SLAT (570-575)	2*
4	D	400880	ANGLE BRACKET WITH HOLE	1*
5	E, F	701035	SQUARE INSERT W/BUTTON	1*
NOTES				
Note	Description			
A	Part #'s <b>226101</b> , <b>701035</b> & <b>501060</b> MUST be ordered together			
B	Older versions identified in note C & D are NOT available. This frame can NOT be retrofitted.			
C	Chair models WITHOUT an "A" or "B" in the serial # (EXAMPLE: 572555555)			
D	Chair models WITH an "A" in the serial # AND made PRIOR to December 1st, 2008 (EXAMPLE: 573 <b>A</b> 555555)			
E	Chair models WITH an "A" in the serial # AND made AFTER to December 1st, 2008 (EXAMPLE: 573 <b>A</b> 555555)			
F	Chair models WITH a "B" in the serial # (EXAMPLE: 2572 <b>B</b> 555555)			


# 2572/2573 Blood Drawing Chair Parts List

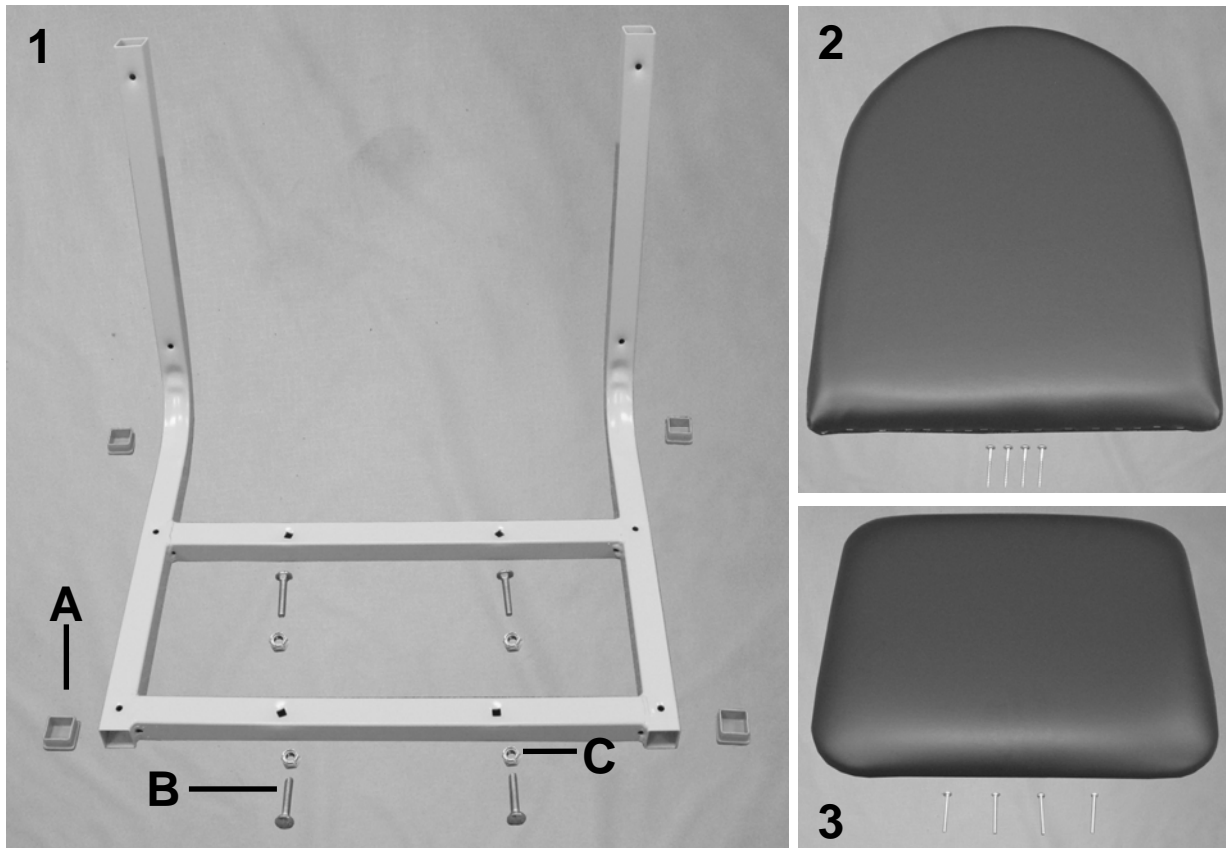

PARTS LIST				
Image Number	Note	Part Number	Description	QTY per Product
1		<b>226300K</b>	<b>SEAT/BACK FRAME ASSM KIT (572 STYLE)</b>	1
1	B	226300	572/573 SEAT & BACK FRAME STL	1
1A	B	700700	SQ 1" CAP, GRAY	4
1B	B	566401	BOLT-CARRIAGE 1/4-20 X 1 1/2 ZINC	4
1C	B	566204	NUT-1/4-20 HEX NYLON LOCKNUT ZINC	4
2	A	U0250XX	572/573 UPHOLSTERED BACK	1
2		566536	SCREW-TAP 10 X 1 3/4 RND STD WASHER SQ T/S ZINC	4
3	A	U0350XX	572/573 UPHOLSTERED SEAT	1
3		566515	SCREW-MACH 10-24 X 1 3/4 PAN PHILLIP M/S ZINC	4
NOTES				
Note	Description			
A	XX- Use color code for specific part number i.e. 02 - Royal Blue			
B	It is NOT recommended that these parts be sold separately however in special cases they can be.			
OPTIONS	For TB133 Add TB after Part Number i.e. U020702 <b>TB</b>			

# 2572/2573 Blood Drawing Chair Parts List

*For "A" and "B" model chairs*

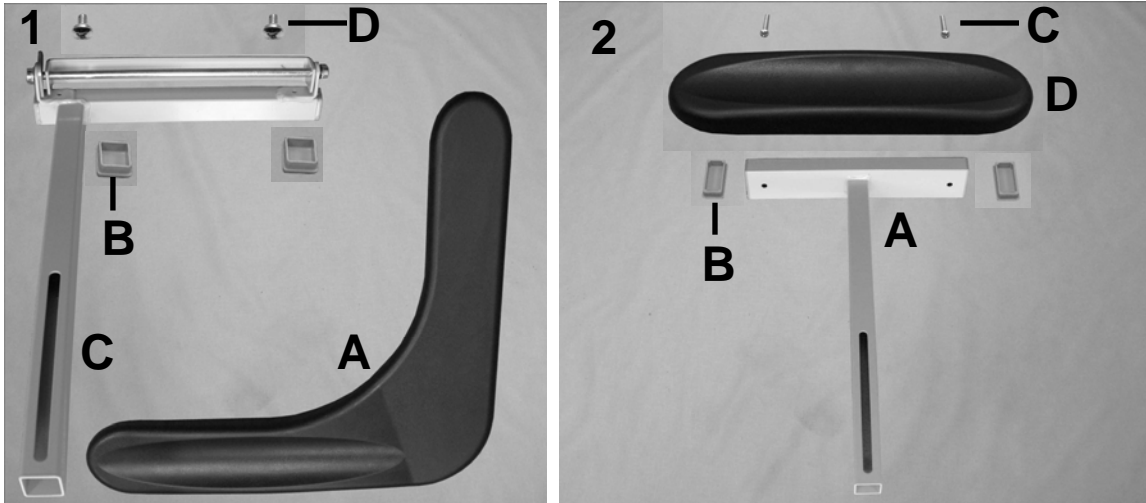


## PARTS LIST

Image	Note	Part Number	Description	QTY per
1	A, B,	<b>226270</b>	<b>WELDED ARM SUPPORT ASSM KIT 570,571,572,573,574,575</b>	1
1B	A,B,D	700700	SQ 1" CAP, GRAY	2
1C	A,B,D	226215	WELDED HINGED ARM SUPPORT	1
1C	A,B,D	402757	STOP-BRACKET, L-SHAPED ARM (570 style)	1
1C	A,B,D	226218	HINGE - BLOOD CHAIR ARM	1
1C	A,B,D	243270	5/16" SS ROD FOR SIDE TABLE	1
1C	A,B,D	500750	5/16" PUSHNUT FASTENER NICKEL	2
1D	A, B	566557	1/4-20 X 3/8" PAN HEAD PHILLIPS MACHINE SCREW	2
1A	A, B	708010	L-ARM, URETHANE, BLOOD CHAIR	1
2	C	<b>226255K</b>	<b>WELDED ARM COLUMN ASSM. SM URETH ARM KIT</b>	1
2A	C	226255	WELDED ARM COLUMN ASSM. SM URETH ARM	1
2B	C, D	701010	1X2X18 RECT INSERT CAP/GLIDE LT.GRAY	2
2C	C	566505	SCREW-MACH 1/4-20 X 1 1/4 RND PHILLIP M/S ZINC	2
2D	C	708020	ARMREST URETHANE, BLOOD CHAIR	1
Note	Description			
A	Used on both models - <b>572 &amp; 573</b>			
B	For <b>ALL</b> chair models WITH an "A" or "B" in the serial # (EXAMPLE: 572 <b>A</b> 555555)			
C	For <b>573</b> chair models WITH an "A" or "B" in the serial # (EXAMPLE: 572 <b>A</b> 555555)			
D	It is NOT recommended that these parts be sold seperately however in special cases they can be.			
	<u>Left &amp; Right is determined - as if sitting in the chair</u>			

# 2572/2573 Blood Drawing Chair Parts List

*For "A" and "B" model chairs*

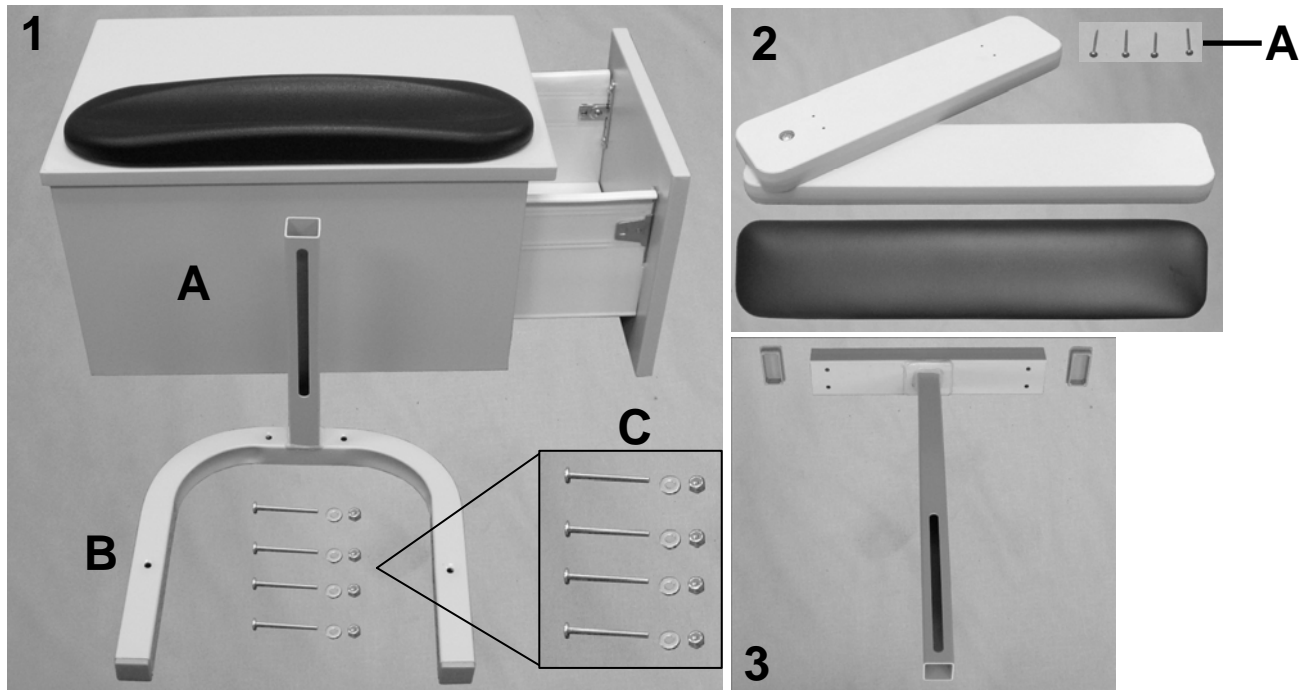

## PARTS LIST


Image	Note	Part Number	Description	QTY per
1	B,C,E	267450	<b>CABINET KIT FOR BLOOD CHAIRS</b>	1
1A	A,B,C,F	267405	<b>CABINET ASSM. COMPLETE (570 STYLE)</b>	1
1A	G	708020	ARMREST URETHANE, BLOOD CHAIR	1
	G	566505	SCREW-MACH 1/4-20 X 1 1/4 RND PHILLIP M/S ZINC	2
1C	G	566516	SCREW-MACH 10-24 X 2 PAN PHILLIP M/S ZINC	4
1C	G	566302	WASHER-#10 SAE F/W ZINC	4
1C	G	566203	NUT-10-24 HEX NYLON LOCK NUT ZINC	4
1B	B,C,F	226340K	<b>WELDED CABINET SUPPORT ASSM. KIT</b>	1
1B	G	226340	WELDED CABINET SUPPORT ASSM.	1
1B	G	700700	SQ 1" CAP, GRAY	2
2	D,F,H	U2101XX	<b>UPHOLSTERED PIVOT ARM w/WOODSET</b>	1
2A	G	566536	SCREW-TAP 10 X 1 3/4 RND STD WASHER SQ T/S ZINC	4
3	D,F	226360K	<b>WELDED PIVOT ARM COLUMN KIT</b>	1
3	G	226360	WELDED PIVOT ARM COLUMN HD (574-575 STYLE)	1
3	G	701010	1X2X18 RECT INSERT CAP/GLIDE LT.GRAY	2

## NOTES

Note	Description
A	572 Chair models with "A" in the serial # made BEFORE June 06, 2007 (EXAMPLE: 572A555555)
B	572 Chair models with "A" in the serial # made ON or AFTER June 06, 2007 (EXAMPLE: 572A555555)
C	572 Chair models with "B" in the serial # (EXAMPLE: 572B555555)
D	572 or 573 chair models WITH an "A" or "B" in the serial #. (EXAMPLE: 572B555555)
E	This KIT includes everything shown in image 1.
F	Can be ordered as a stand-alone item
G	It is NOT recommended that these parts be sold seperately however in special cases they can be.
H	XX- Use color code for specific part number i.e. 02 - Royal Blue
OPTIONS	For TB133 Add TB after Part Number i.e. U210102TB
	<i>Left &amp; Right is determined - as if sitting in the chair</i>

# 2572/2573 Blood Drawing Chair Parts List

For chair models WITHOUT "A" or "B"

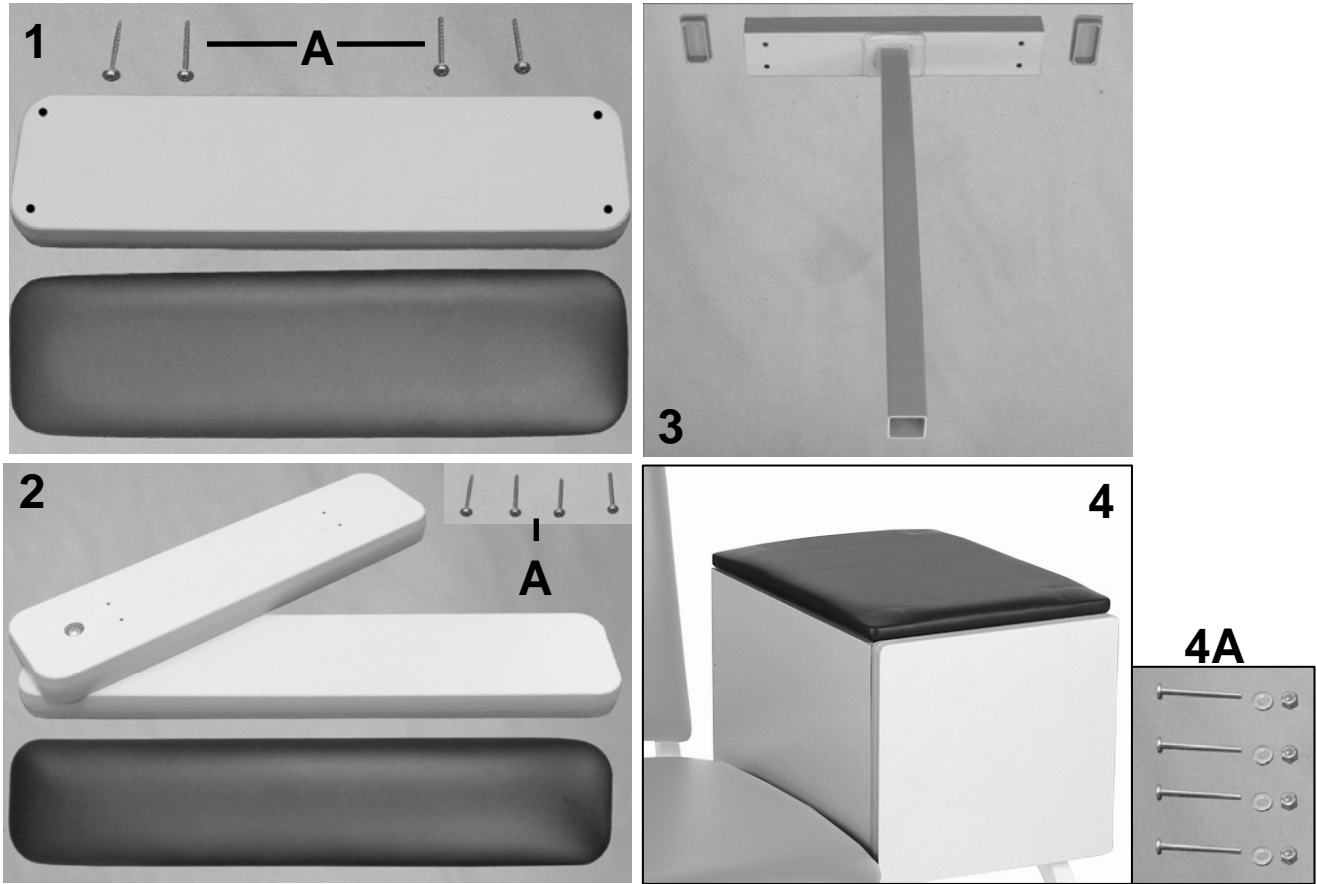

PARTS LIST				
Image	Note	Part Number	Description	QTY per
1	A,D,F	U2001XX	UPHOLSTERED STATIONARY ARM w/WOODSET	1
1A	C	566536	SCREW-TAP 10 X 1 3/4 RND STD WASHER SQ T/S ZINC	4
2	A,B,D,F	U2101XX	UPHOLSTERED SWIVEL ARM w/WOODSET	1
2A	C	566536	SCREW-TAP 10 X 1 3/4 RND STD WASHER SQ T/S ZINC	4
3	A,B,E	226200	WELDED ARM COLUMN, NO SLOT (570 THRU 575 & WOOD ARMS)	1
3	A,B,E	701010	1X2X18 RECT INSERT CAP/GLIDE LT.GRAY	2
4	B,D,F	U2301XX	UPHOLSTERED CABINET PAD w/CABINET	1
4A	C	566516	SCREW-MACH 10-24 X 2 PAN PHILLIP M/S ZINC	4
4A	C	566302	WASHER-#10 SAE F/W ZINC	4
4A	C	566203	NUT-10-24 HEX NYLON LOCK NUT ZINC	4
NOTES				
Note	Description			
A	For <b>573</b> chair models WITHOUT an "A" or "B" in the serial # ONLY (EXAMPLE: 57255555)			
B	For <b>572</b> chair models WITHOUT an "A" or "B" in the serial # ONLY (EXAMPLE: 57255555)			
C	It is NOT recommended that these parts be sold seperately however in special cases they can be.			
D	XX- Use color code for specific part number i.e. 02 - Royal Blue			
E	Parts MUST be ordered together - See ALL parts referencing NOTE "E"			
F	Uph. items are NOT available without the woodset or cabinet.			
OPTIONS	For TB133 Add TB after Part Number i.e. U200102 <b>TB</b>			
	<i>Left &amp; Right is determined - as if sitting in the chair</i>			

For additional options please see:

**IV POLE OPTIONS PAGE**