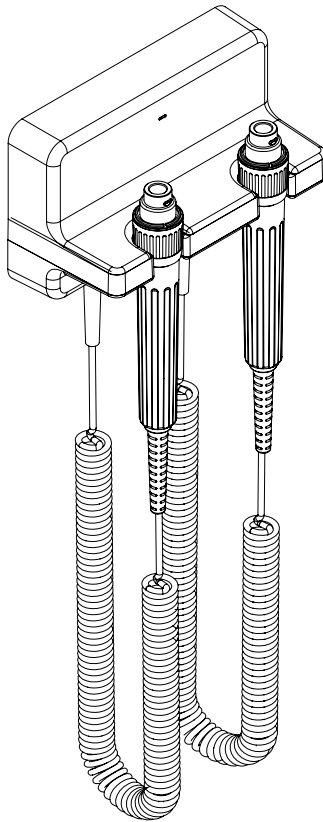
Instructions for Use
Diagnostic Station

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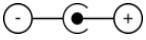
Amico

Intended Use

This product is intended to supply power to Amico Diagnostic 3.5V instruments.

Symbol Descriptions

Positive Polarity.

Separate Waste Electronic and Electrical Equipment (WEEE) from other disposables. Prepare this product for reuse or separate collection as specified by the WEEE Directive 2002/96/EC or local law.

Type BF applied part.

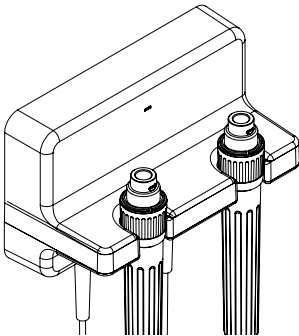
Consult instructions for use.


General Warnings and Cautions

- ⚠ WARNING:** Read and understand the instructions prior to using this product. Failure to understand the operating requirement for the product can result in injury to you or the patient and can damage the product.
- ⚠ WARNING:** Explosion hazard. Do not operate this product in the presence of flammable anesthetics or another flammable substance in combination with air, oxygen enriched environments, or nitrous oxide.
- ⚠ WARNING:** Electric shock hazard. Do not simultaneously touch the patient and the center metal contact of the handle.
- ⚠ WARNING:** Do not modify this equipment. Any modification of this equipment can lead to patient injury. Any modification of this equipment voids the product warranty.
- ⚠ CAUTION:** Only authorized personnel by Amico Diagnostic Incorporated can perform service on this product.
- ⚠ CAUTION:** Make sure that the Diagnostic Station can be unplugged from the AC power source, never block access to the wall socket.
- ⚠ CAUTION:** Rx only (for professional use).

Included Hardware

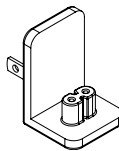1. Base Station

2. Socket Adapter

3. Power Supply

4. Screws

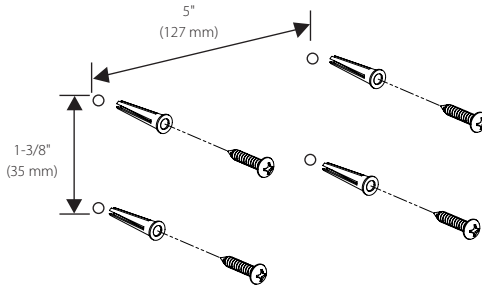
5. Wall Anchors


Pre-Installation Information (Please Read Carefully Before Starting)

⚠ WARNING: Amico Diagnostic Incorporated is not responsible for the integrity of any mounting installation. Amico Diagnostic Incorporated recommends that customers contact their Biomedical Engineering Department or maintenance service to ensure professional installation for safety and reliability of any mounting accessory.


1. Tape the provided Diagnostic Station outline to wall in the desired location.
2. At 4 locations marked \oplus , drill a 3/16" (4.75 mm) hole to a depth of at least 1.5" (38 mm).
3. Remove Diagnostic Station outline and fully insert anchors into drilled holes.
4. Thread screws into anchors leaving 1/8" (3 mm) of exposed threads.

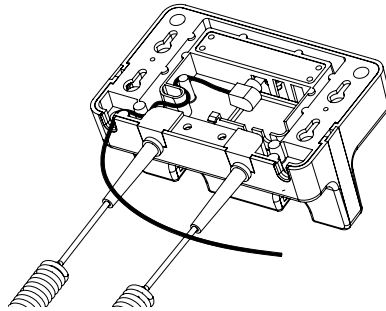5. Plug barrel connector of power supply into base station. Cord length can be adjusted by using the integrated cord wrap on the back of unit.

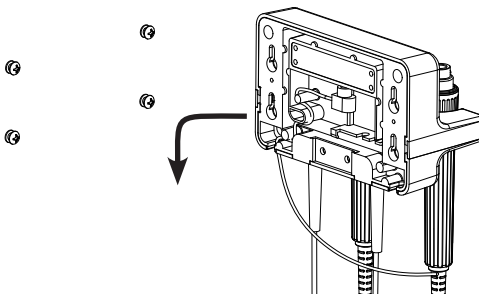
6. Mount the base station by fitting the holes on the back of the unit over the screw heads. Push downward on the base station to secure.

Setup

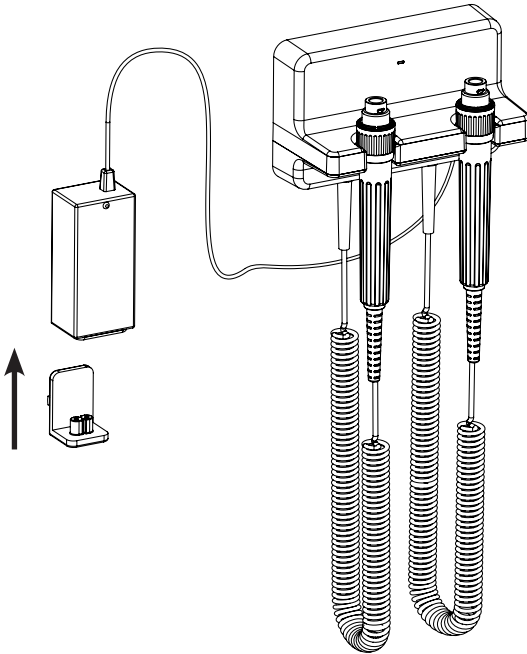7. Connect the socket adapter to the power supply. Plug the power supply into the wall socket.

8. The LED on the base station will illuminate, indicating that the power is on.

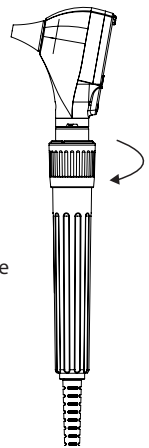
Using the Handles

1. Remove the handle from the cradle.
2. Adjust the light intensity by turning the rheostat on the handle:
 - Turning clockwise increases the light intensity.
 - Turning counterclockwise decreases the light intensity.
3. Refer to the directions for the instrument head when examining a patient.

⚠ CAUTION: To prevent damage, do not overstretch the cords on the handles.

⚠ CAUTION: To minimize the external housing temperature of the instrument head, the On-Time must not to exceed 2 minutes, and the Off-Time must be a minimum of 10 minutes.

4. Return the handle to the cradle to turn handle off.
5. The handle will automatically turn off within 5 minutes to protect the bulb if accidentally left on. Returning the handle to the cradle will reset the timer.

Cleaning

⚠ WARNING: Unplug power supply from wall socket prior to cleaning the diagnostic station.

⚠ WARNING: DO NOT autoclave the diagnostic station.

⚠ WARNING: Liquids can damage electronics inside the diagnostic station. Prevent liquids from spilling on or dripping into the diagnostic station.

Clean the diagnostic station on a routine basis according to your facility's protocols and standards or local regulations.

To clean the diagnostic station (excluding the instrument heads), use any of the following:

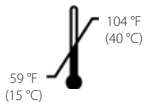
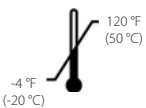
- A clean cloth slightly dampened with 70 percent isopropyl alcohol or 10 percent chlorine bleach solution. Wipe dry with a clean cloth.
- Sani-Cloth Plus.
- CaviWipes.

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